microson

MU-0012-001-LP2 Rev. E 2014-03-24

Printed in Spain

M-34 Digital BTE

AGC / HIGHPOWER / PP / PP-AGC / STD / FC OPEN

MANUAL DE USO

ES

USER'S MANUAL

EN

MANUALE D'USO

IT

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PT

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FR

MICROSON S.A.
Pere IV, 160
08005 BARCELONA, SPAIN
Tel. +34 93 300 58 00
www.microson.es

Ref. 91360

microson

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Congratulations and welcome!

You have bought a MICROSON hearing aid featuring the latest digital technology.

Your hearing device is the result of years of research and development and is designed to make sure you enjoy all the sounds of life.

The team at Microson would like to wish you great success with it.

Intended use	16
Disposal Information	16
How it works	17
Explanation of symbols	18
Safety Instructions	19
Description of your hearing aid	20
Battery (insertion and changing)	21
Volume adjustment	22
Switching the instrument ON/OFF	22
Switch position	23
Changing the adapter (Thin Tube)	24
Care and maintenance of your hearing aid	25
Environmental operating conditions	25
Replacements	26
Troubleshooting	27

INTENDED USE

Microson hearing aids are designed for correcting hearing problems and compensating for hearing loss by selective sound amplification. Your hearing aids should be fitted by a hearing care specialist.

For safe use, read the instructions in this manual carefully.

This manual was drawn up in compliance with the EN 1041 and EN 980 standards.


DISPOSAL INFORMATION

This product must not be disposed together with the tipstic waste. All users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner. Remove the batteries before disposing of the equipment. Do not dispose of old batteries with your household waste, but dispose of them at a battery collection station at a recycling site or in a shop. Consult your municipal authority or your dealer for information about disposal.

Within the European Union, the marked equipment is covered by "Directive 2002/96/EC of the European Paliament and of the Council of 27 January 2003 on waste electrical andelectronical equipment."Amended by "Directive 2003/108/EC" (WEEE).

HOW IT WORKS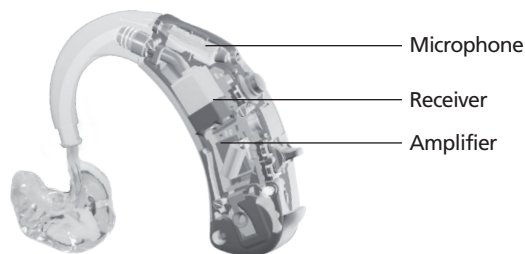


The BTE (Behind the Ear BTE) hearing aids are anatomical instruments placed behind the ear. Are powered by a battery and are typically designed to cover severe or profound hearing losses. All mechanical and electronic components are contained in an ergonomic housing. They adapt to the user's ear by molds which leaves a conductive tube that attaches to the side of the hearing aid.

MICROPHONE: receives the sound and converts it into an electrical signal.

AMPLIFIER: digital processor that receives the signal and adapts it to the parameters introduced for each loss.

RECEIVER: converts the electrical signal back into sound and transmits it into the user's ear.

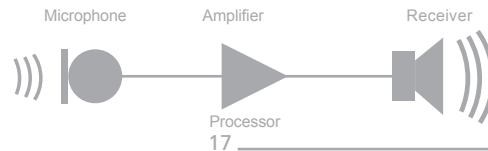This symbol indicates unsafe situations or procedures that may lead to injury or cause damage to your hearing aid or any of its components.

Operating instructions.

This symbol indicates that it is important to follow the advice and warnings contained in this document.

The CE Mark enables Microson to confirm that its hearing aids comply with European Directive 93/42 EEC concerning medical devices.

This symbol indicates that the products described in these user instructions adhere to the requirements for a Type B component under IEC 60601-2-66. The surface of the hearing aid is specified as a Type B component.

The symbol with the crossed-out garbage bin warns you that this hearing aid may not be disposed of with normal household waste. Please return your hearing aid to your hearing healthcare professional or a collection point for safe, environmentally-friendly disposal.

⚠ WARNING!

- It is dangerous to swallow batteries. Keep batteries out of the reach of children.
- Never use your hearing aids in places where there is medical equipment (X-ray, high frequency, magnetic resonance or short-wave therapy treatments).
- Do not use the hearing aids in areas where there is a risk of explosion!
- **M-34 Digital** hearing aids can generate output levels in excess of 132 dB_{SPL}.

⚠ CAUTION!

- Protect your hearing aids against moisture and sources of heat.
- Do not use your hearing aids while showering, swimming or spraying your hair.
- Protect your hearing aids from hard knocks.
- Never use a microwave or other heat emitters to dry your hearing aids.
- Regular use of a dehumidifier is recommended to keep your hearing aid in optimum working conditions: Ask your hearing care specialist.


📖 INFORMATION

- Remove the batteries if you do not intend to use your hearing aids for a long time.
- Replace spent batteries immediately.
- Do not dispose of spent batteries with normal household waste! Take them to the hearing care specialist for proper disposal without harming the environment.

DESCRIPTION OF YOUR HEARING AID

M-34 DIGITAL BTE

- ☐ STD ☐ HIGHPOWER ☐ FC OPEN
☐ AGC ☐ PP-AGC ☐ PP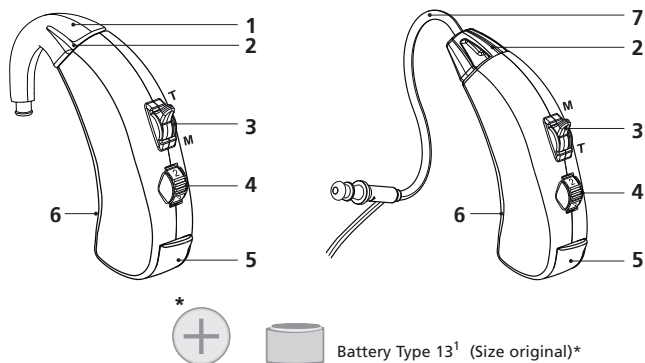

1. Ergonomic ear-hook
2. Microphone
3. M-T Switch
4. Volume Control
5. Battery compartment*
6. Trimmers
(For use exclusively by the Hearing Aid specialist)
7. Thin Tube

¹IEC 60086 Designation: PR48


BATTERY (INSERTION AND REMOVAL)

1- Opening the battery compartment.


2- Insert the battery with the "+" symbol facing upwards and close the battery compartment.

If the battery is not correctly inserted, the hearing aid will not function.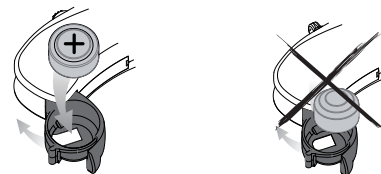

3- To remove the battery, face the opening of the hearing aid downwards, open the battery compartment and allow it to fall into your hand.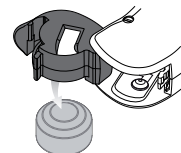

REMEMBER THAT YOUR HEARING AID WILL WARN YOU VIA THREE AUDIBLE TONES 🎵 WHEN THE BATTERY IS DEAD.


The volume control allows you to adjust the amplification of your hearing aid.

Use a finger to rotate the volume control wheel, as indicated in the figure.

Number 1 is the lowest level.
Number 4 is the highest level.


SWITCHING ON / OFF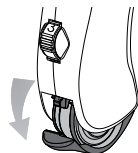

DISCONNECT

TURNING OFF:

To turn off your hearing aid, partially open the battery compartment.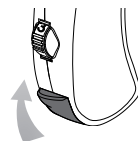

CONNECT

TURNING ON:

To turn on your hearing aid, close the battery compartment.

Your hearing aid has 2-position (T-M) switch for:

Switch Position T:

Program designed for talking on the phone or for use in public places such as theatres and churches with induction loop. (Consult your specialist).
You will hear one tone ♪ when you select this program.

Switch Position M:


Normal hearing program.

This program is recommended for mobile phone* use.

You will hear two tones ♪♪ when you select this program.


M-34 DIGITAL	
STD / HIGHPOWER / AGC / PP-AGC / PP	FC OPEN

*Your hearing aid complies with the IEC 60118-13 standard on electromagnetic compatibility.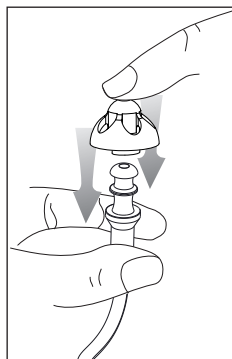


INSERTING THE ADAPTER:

Hold the tube firmly and push the adapter downwards.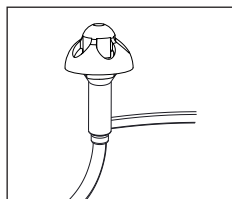

Make sure that the adapter is completely attached to the tube.

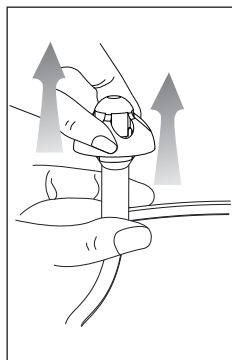EXTRACTING THE ADAPTER:

Hold the tube firmly and pull the adapter upwards until the two parts become separated.

INFORMATION

- Clean the moulds frequently using a special towelette. We recommend using MICROCLEAN (ask your hearing care specialist).
- Ask for MICRODRY to prevent the damaging effects of moisture.
- Always have a spare battery to hand.
- ⚠ **ATTENTION!**
- Never clean your hearing aids with alcohol, peroxide or detergents. Use a soft, damp cloth.

Do not get your hearing aids wet!

TIPS / ADAPTERS

Cerumen (ear wax) in the tips / adapters or moulds affects sound quality.

Clean the tips / adapters or moulds immediately after removing the receiver with a soft cloth. No other tools are required.


ENVIRONMENTAL OPERATING CONDITIONS

For optimum operation we recommend using your hearing aids within a range of 0 / 40 °C (degrees Celsius). Likewise, the relative humidity during storage should not exceed 95%.

TIPS / ADAPTERS (FOR THIN TUBE)

83414	Open 4 mm	
83407	Open 6 mm	
83419	Open 8 mm	
83420	Open 10 mm	
83408	Double closed 8-10 mm	
83413	Double closed 10-12 mm	
83412	Closed 8 mm	
91189	Semi-open	

RECOMMENDED BATTERIES

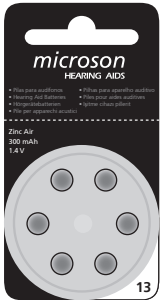Ref. 82876 Microson Battery Type 13
(IEC 60086: PR48)

SYMPTOM	POSSIBLE CAUSE	SOLUTION
The hearing aid does not work	Worn battery defective hearing aid	Replace battery consult your specialist
Insufficient volume	Mould blocked or obstructed	Eliminate obstruction and clean the mould
Sound distorted or unclear	Worn battery	Replace battery

Manufacturer's customer service:  +34 93 300 58 00
 helpdesk@microson.es

 NOTE

For any other problems or doubts, contact your audioprosthologist or retailer.

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