microson

MU-0012-002-LP3 Rev. E 2014-01-14

Printed in Spain

M-34 Digital BTE

AGC / HIGHPOWER / PP / PP-AGC / STD / FC OPEN

USER'S MANUAL

EN

KULLANIM KILAVUZU

TR

MANUAL DE UTILIZARE

RO

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microson

Congratulations and welcome!

You have bought a MICROSON hearing aid featuring the latest digital technology.

Your hearing device is the result of years of research and development and is designed to make sure you enjoy all the sounds of life.

The team at Microson would like to wish you great success with it.

Intended use	2
Disposal Information	2
How it works	3
Explanation of symbols	4
Safety Instructions	5
Description of your hearing aid	6
Battery (insertion and changing)	7
Volume adjustment	8
Switching the instrument ON/OFF	8
Switch position	9
Changing the adapter (Thin tube)	10
Care and maintenance of your hearing aid	11
Environmental operating conditions	11
Replacements	12
Troubleshooting	13

INTENDED USE

Microson hearing aids are designed for correcting hearing problems and compensating for hearing loss by selective sound amplification. Your hearing aids should be fitted by a hearing care specialist.

For safe use, read the instructions in this manual carefully.

This manual was drawn up in compliance with the EN 1041 and EN 980 standards.


DISPOSAL INFORMATION

This product must not be disposed together with the domestic waste. All users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner. Remove the batteries before disposing of the equipment. Do not dispose of old batteries with your household waste, but dispose of them at a battery collection station at a recycling site or in a shop. Consult your municipal authority or your dealer for information about disposal.

Within the European Union, the marked equipment is covered by "Directive 2002/96/EC of the European Parliament and of the Council of 27 January 2003 on waste electrical and electronic equipment." Amended by "Directive 2003/108/EC" (WEEE).

HOW IT WORKS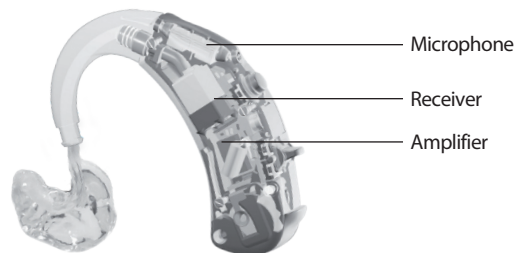


The BTE (Behind the Ear BTE) hearing aids are anatomical instruments placed behind the ear. They are powered by a battery and are typically designed to cover severe or profound hearing losses. All mechanical and electronic components are contained in an ergonomic housing. They adapt to the user's ear by molds which leaves a conductive tube that attaches to the side of the hearing aid.

MICROPHONE: receives the sound and converts it into an electrical signal.

AMPLIFIER: digital processor that receives the signal and adapts it to the parameters introduced for each loss.

RECEIVER: converts the electrical signal back into sound and transmits it into the user's ear.

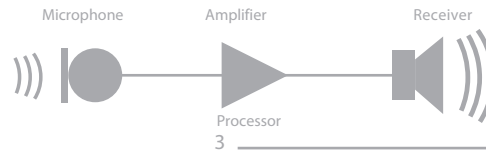This symbol indicates unsafe situations or procedures that may lead to injury or cause damage to your hearing aid or any of its components.

Operating instructions.

This symbol indicates that it is important to follow the advice and warnings contained in this document.

The CE Mark enables Microson to confirm that its hearing aids comply with European Directive 93/42 EEC concerning medical devices.

This symbol indicates that the products described in these user instructions adhere to the requirements for a Type B component under IEC 60601-2-66. The surface of the hearing aid is specified as a Type B component.

The symbol with the crossed-out garbage bin warns you that this hearing aid may not be disposed of with normal household waste. Please return your hearing aid to your hearing healthcare professional or a collection point for safe, environmentally-friendly disposal.

WARNING!

- It is dangerous to swallow batteries. Keep batteries out of the reach of children.
- Never use your hearing aids in places where there is medical equipment (X-ray, high frequency, magnetic resonance or short-wave therapy treatments).
- Do not use the hearing aids in areas where there is a risk of explosion!
- **M-34 Digital** hearing aids can generate output levels in excess of 132 dB_{SPL}.

CAUTION!


- Protect your hearing aids against moisture and sources of heat.
- Do not use your hearing aids while showering, swimming or spraying your hair.
- Protect your hearing aids from hard knocks.
- Never use a microwave or other heat emitters to dry your hearing aids.

INFORMATION

- Remove the batteries if you do not intend to use your hearing aids for a long time.
- Replace spent batteries immediately.
- Do not dispose of spent batteries with normal household waste! Take them to the hearing care specialist for proper disposal without harming the environment.

M-34 DIGITAL BTE

- ☐ STD ☐ HIGHPOWER ☐ FC OPEN
☐ AGC ☐ PP-AGC ☐ PP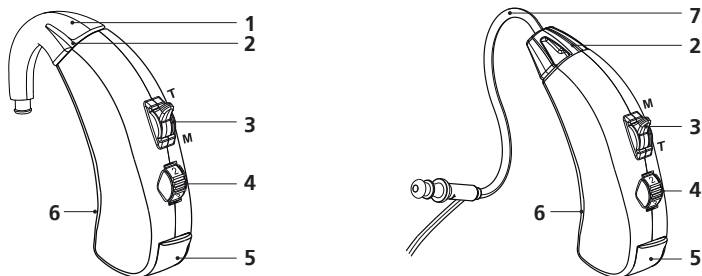

Battery Type 13¹ (Size original)*


1. Ergonomic ear-hook
2. Microphone
3. M-T Switch
4. Volume Control
5. Battery compartment*
6. Trimmers
(For use exclusively by the Hearing Aid specialist)
7. Thin Tube

1- Opening the battery compartment.


2- Insert the battery with the “+” symbol facing upwards and close the battery compartment.

 If the battery is not correctly inserted, the hearing aid will not function.

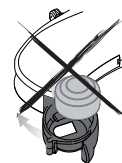
3- To remove the battery, face the opening of the hearing aid downwards, open the battery compartment and allow it to fall into your hand.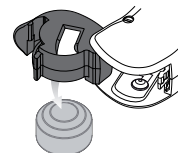

REMEMBER THAT YOUR HEARING AID WILL WARN YOU VIA THREE AUDIBLE TONES 🎵🎵🎵 WHEN THE BATTERY IS DEAD.

¹IEC 60086 Designation: PR48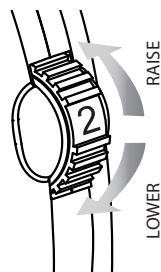


The volume control allows you to adjust the amplification of your hearing aid.

Use a finger to rotate the volume control wheel, as indicated in the figure.

Number 1 is the lowest level.
Number 4 is the highest level.


SWITCHING ON / OFF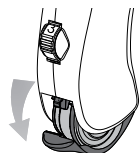

DISCONNECT

TURNING OFF:

To turn off your hearing aid, partially open the battery compartment.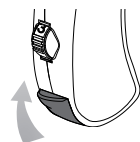

CONNECT

TURNING ON:

To turn on your hearing aid, close the battery compartment.

Your hearing aid has 2-position (T-M) switch for:

Switch Position T:

Program designed for talking on the phone or for use in public places such as theatres and churches with induction loop. (Consult your specialist).

You will hear one tone ♪ when you select this program.

Switch Position M:


Normal hearing program.

This program is recommended for mobile phone* use.

You will hear two tones ♪♪ when you select this program.


M-34 DIGITAL	
STD / HIGHPOWER / AGC / PP-AGC / PP	FC OPEN

*Your hearing aid complies with the IEC 60118-13 standard on electromagnetic compatibility.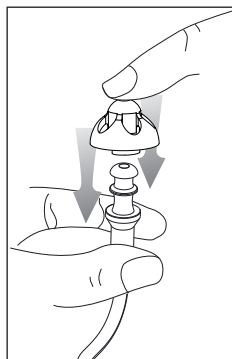


INSERTING THE ADAPTER:

Hold the tube firmly and push the adapter downwards.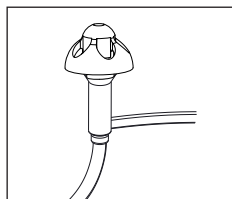

Make sure that the adapter is completely attached to the tube.

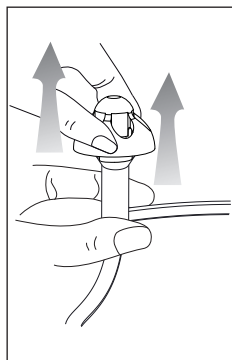EXTRACTING THE ADAPTER:

Hold the tube firmly and pull the adapter upwards until the two parts become separated.

INFORMATION

- Clean your hearing aid regularly, using specially designed wipes. (Consult your specialist).
- Regular use of a dehumidifier is recommended to keep your hearing aid in optimum working conditions. (Consult your specialist).
- Always have a spare battery available.

ATTENTION!

- Do not clean your hearing aid with alcohol, hydrogen peroxide or detergents: use a soft, moist cloth.

Do not wash your hearing aid!

TIPS / ADAPTERS

Cerumen (ear wax) in the tips / adapters or moulds affects sound quality.

Clean the tips / adapters or moulds immediately after removing the receiver with a soft cloth. No other tools are required.


ENVIRONMENTAL OPERATING CONDITIONS

For optimum operation we recommend using your hearing aids within a temperature range of 0 / 40 °C (degrees Celsius). Likewise, the relative humidity during storage should not exceed 95%.

TIPS / ADAPTERS (FOR THIN TUBE)

83414	Open 4 mm	
83407	Open 6 mm	
83419	Open 8 mm	
83420	Open 10 mm	
83408	Double closed 8-10 mm	
83413	Double closed 10-12 mm	
83412	Closed 8 mm	
91189	Semi-open	

RECOMMENDED BATTERIES:

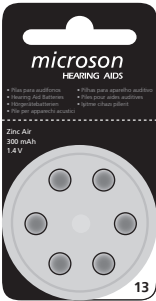Ref. 82876 Microson Battery Type 13 (IEC 60086: PR48)

SYMPTOM	POSSIBLE CAUSE	SOLUTION
The hearing aid does not work	Worn battery defective hearing aid	Replace battery consult your specialist
Insufficient volume	Tip/ adapter or Mould blocked or obstructed	Remove the obstruction and clean the tip / adapter or mould
Sound distorted or unclear	Worn battery	Replace battery

Manufacturer's
customer service:

+34 93 300 58 00
helpdesk@microson.es

NOTE

For any other problems or doubts, contact your audioprosthologist or retailer.

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