microson

MU-0012-004-LP3 RevA 2014-07-14

M-34 Digital

M-34 DIGITAL FRONT BTE

Printed in Spain

USER'S MANUAL

EN

KULLANIM KILAVUZU

TR

MANUAL DE UTILIZARE

RO

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microson

Congratulations and welcome!

You have bought a MICROSON hearing aid featuring the latest digital technology.

Your hearing device is the result of years of research and development and is designed to make sure you enjoy all the sounds of life.

The team at Microson would like to wish you great success with it.

Intended use	2
Disposal Information	2
How it works	3
Explanation of symbols	4
Safety Instructions	5
Description of your hearing aid	6
Battery (insertion and changing)	7
Volume adjustment	8
Switching the instrument ON/OFF	8
Battery Lock	9
Programs push button	10
Audio input	10
Audio Shoe (insertion, extraction)	11
Care and maintenance of your hearing aid	12
Environmental operating conditions	12
Troubleshooting	13

INTENDED USE

Microson hearing aids are designed for correcting hearing problems and compensating for hearing loss by selective sound amplification. Your hearing aids should be fitted by a hearing care specialist.

For safe use, read the instructions in this manual carefully.

This manual was drawn up in compliance with the EN 1041 and EN 980 standards.


DISPOSAL INFORMATION

This product must not be disposed together with the domestic waste. All users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner. Remove the batteries before disposing of the equipment. Do not dispose of old batteries with your household waste, but dispose of them at a battery collection station at a recycling site or in a shop. Consult your municipal authority or your dealer for information about disposal.

Within the European Union, the marked equipment is covered by "Directive 2002/96/EC of the European Parliament and of the Council of 27 January 2003 on waste electrical and electronic equipment." Amended by "Directive 2003/108/EC" (WEEE).

HOW IT WORKS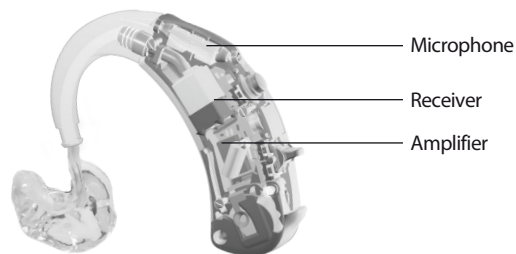


The BTE (Behind the Ear BTE) hearing aids are anatomical instruments placed behind the ear. They are powered by a battery and are typically designed to cover severe or profound hearing losses. All mechanical and electronic components are contained in an ergonomic housing. They adapt to the user's ear by molds which leaves a conductive tube that attaches to the side of the hearing aid.

MICROPHONE: receives the sound and converts it into an electrical signal.

AMPLIFIER: digital processor that receives the signal and adapts it to the parameters introduced for each loss.

RECEIVER: converts the electrical signal back into sound and transmits it into the user's ear.

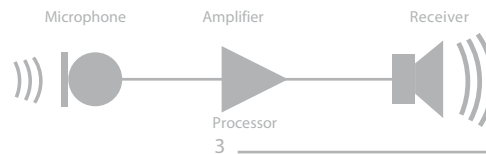Indicates situations where you may be injured.
Indicates unsafe situations or procedures that may lead to injury or damage to your hearing aid.
Indicates possible damage or alteration of the hearing aid or any of its components.

Operating instructions.
Indicates that it is important to follow the advice and warnings contained in this document.

Thanks to CE marking, the manufacturer confirms that its hearing aids complies with European Directive 93/42 EEC concerning medical devices.

This symbol indicates that the products described in these user instructions adhere to the requirements for a Type B component under IEC 60601-2-66. The surface of the hearing aid is specified as a Type B component.

The symbol with the crossed-out garbage bin warns you that this hearing aid may not be disposed of with normal household waste. Please return your hearing aid to your hearing healthcare professional or a collection point for safe, environmentally-friendly disposal.

⚠ WARNING!


- It is dangerous to swallow batteries. Keep batteries out of the reach of children.
- Never use your hearing aids in places where there is medical equipment (X-ray, high frequency, magnetic resonance or short-wave therapy treatments).
- Do not use the hearing aids in areas where there is a risk of explosion!
- **M-34 Digital Front BTE** hearing aids can generate output levels in excess of 132 dBSPL.

⚠ CAUTION!

- Protect your hearing aids against moisture and sources of heat.
- Do not use your hearing aids while showering, swimming or spraying your hair.
- Protect your hearing aids from hard knocks.
- Never use a microwave or other heat emitters to dry your hearing aids.
- Regular use of a dehumidifier is recommended to keep your hearing aid in optimum working conditions: Ask your hearing care specialist.

📖 INFORMATION


- Remove the batteries if you do not intend to use your hearing aids for a long time.
- Replace spent batteries immediately.
- Do not dispose of spent batteries with normal household waste! Take them to the hearing care specialist for proper disposal without harming the environment.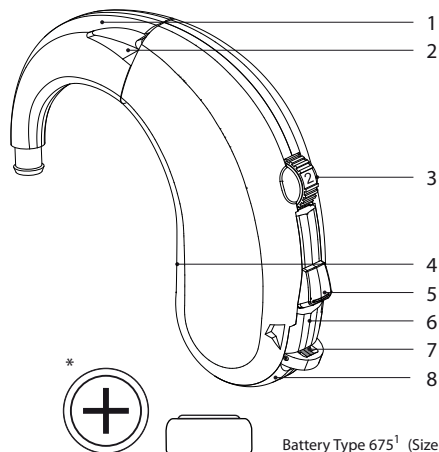

1. Ergonomic-ear-hook design (Standard / Pediatric)
2. Microphone
3. Volume Control
4. Trimmers
(For use exclusively by the Hearing Aid specialist)
5. Programs push button
6. Audio Input
7. Battery Lock
8. Battery compartment



¹IEC 60086 Designation: PR44

1- Opening the battery compartment.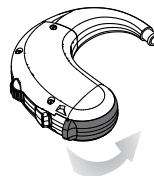


2- Insert the battery with the “+” symbol facing upwards and close the battery compartment.

 If the battery is not correctly inserted, the hearing aid will not function.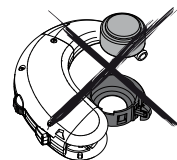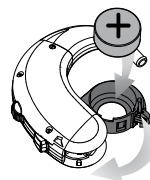

3- To remove the battery, face the opening of the hearing aid downwards, open the battery compartment and allow it to fall into your hand

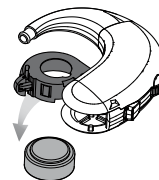REMEMBER THAT YOUR HEARING AID WILL WARN YOU VIA THREE AUDIBLE TONES 🎵 WHEN THE BATTERY IS DEAD

The volume control allows you to adjust the amplification of your hearing aid.

Use a finger to rotate the volume control wheel, as indicated in the figure.

Number 1 is the lowest level.
Number 4 is the highest level.

SWITCHING ON / OFF

DISCONNECT

TURNING OFF:


To turn off your hearing aid, partially open the battery compartment.

CONNECT

TURNING ON:

To turn on your hearing aid, close the battery compartment.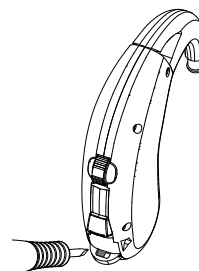


Your hearing aid is equipped with a battery holder locking system. This lock prevents undesired accidental opening of the compartment.


It is also recommended when the hearing aid is for a child, because removal and replacement of the battery must be done under adult supervision. This element is located in the upper portion of the opening tab of the battery compartment.

To prevent opening, insert a small screwdriver and slide it to one side.

Unlocking it can only be done by sliding the safety back to its original position with the help of a small screwdriver.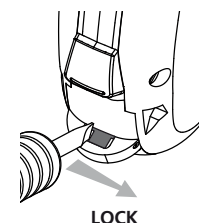


LOCK

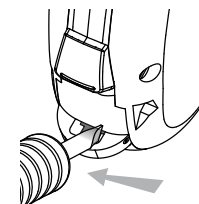UNLOCK

PROGRAMS PUSH BUTTON

Your hearing aid has a push button that enables you to select the desired program.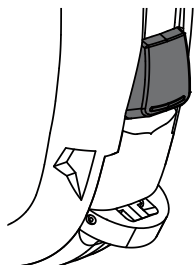

Position M: Normal hearing program.

This program is recommended for mobile phone¹ use.

You will hear 1 tone ♪ when you select this program.

Position T: Program designed for talking on the phone or for use in public places such as theatres and churches with induction loop. (Consult your specialist). You will hear 2 tones ♪♪ when you select this program.

¹Your hearing aid complies with the IEC 60118-13 standard on electromagnetic compatibility.

AUDIO INPUT

Your hearing aid has a built-in audio input jack. This allows you to connect your hearing aid to other audio equipment with FM system, television or various audio equipment. It transmits the source sound¹ directly to your hearing aid, which keeps the signal clear and free of external interference. Only use the Audio Shoe designed specifically for the hearing aid.

⚠ WARNING!

Risk of injury by electrical shock!

Connect the audio input only to devices that comply with IEC 60065 (IEC standard for audio, video and other electronic devices).

¹Your hearing aid complies with the IEC 60118-6 norm, which guarantees its compatibility with external electrical and electro-acoustical signal.



AUDIO SHOE

INSERTION

1. Open the battery door until to the first position, lift the top of the connector and close the battery holder again.

2. Place the audio shoe on top, line up the jack contacts with those that are now visible on the hearing aid.

3. Push the audio shoe downward until it couples with the hearing aid.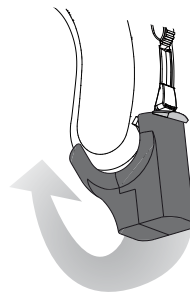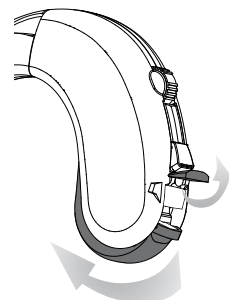


REMOVAL

1. Hold the device with one hand and the shoe with the other.

2. Pull the shoe outwards until it separates from the hearing aid.

3. Close the connector top, hiding the contacts below it.

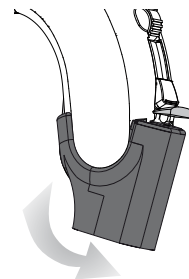 INFORMATION

- Clean your hearing aid regularly, using specially designed wipes. (Consult your specialist).
- Regular use of a dehumidifier is recommended to keep your hearing aid in optimum working conditions. (Consult your specialist).
- Always have a spare battery available.

 ATTENTION!

- Do not clean your hearing aid with alcohol, hydrogen peroxide or detergents: use a soft, moist cloth.

Do not wash your hearing aid!

ENVIRONMENTAL OPERATING CONDITIONS

For optimum operation we recommend using your hearing aids within a temperature range of 0 / 40°C (degrees Celsius). Likewise, the relative humidity during storage should not exceed 95%.

SYMPTOM	POSSIBLE CAUSE	SOLUTION
The hearing aid does not work	Worn battery defective hearing aid	Replace battery consult your specialist
Insufficient volume	Mould blocked or obstructed	Eliminate obstruction and clean the mould
Sound distorted or unclear	Worn battery	Replace battery

Manufacturer's
customer service:

+34 93 300 58 00
helpdesk@microson.es

 NOTE

For any other problems or doubts, contact your audioprosthologist or retailer.

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