microson

MU-0016-002-LP3 Rev. B 2017-04-18

M4RIC
M2RIC

Printed in Spain

USER'S MANUAL

EN

KULLANIM KILAVUZU

TR

MANUAL DE UTILIZARE

RO

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microson

Congratulations and welcome!

You have bought a MICROSON hearing aid featuring the latest digital technology.

Your new hearing aid is the result of years of research and development and is designed to make sure you enjoy all the sounds of life.

The Microson team would like to wish you great success with your hearing aid.

It is important that you read and follow the instructions in this manual to prevent injury or any damage to your hearing aid.

This manual was drawn up in compliance with the EN 1041 and ISO 15223-1 standards.

Intended use	2
Disposal Information	2
How it works	3
Safety Instructions	4-5
Explanation of symbols	6
Description of your hearing aid	7
Volume Adjustment	8
Switching off via the battery compartment	8
Battery (insertion and replacement)	9
Replacing the adapter	10
Programme button	11
Programme selector	11
Changing the earpiece	12-13
Inserting and installing the mould	14
Care and maintenance of your hearing aid	15
Environmental operating conditions	15
Troubleshooting	16
Options	17

INTENDED USE

Microson hearing aids are designed for correcting hearing problems and compensating for hearing loss by selective sound amplification. Your hearing aids should be fitted by a hearing care specialist.

For safe use, read the instructions in this manual carefully.

This manual was drawn up in compliance with the EN 1041 and EN 980 standards.

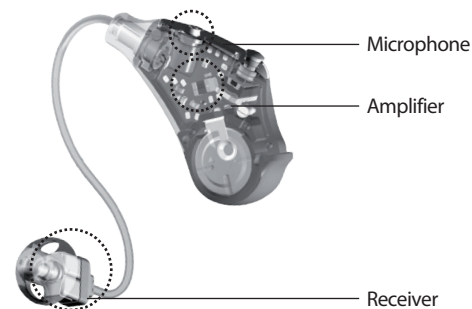
DISPOSAL INFORMATION

This product must not be disposed together with the domestic waste. All users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner. Remove the batteries before disposing of the equipment. Do not dispose of old batteries with your household waste, but dispose of them at a battery collection station at a recycling site or in a shop. Consult your municipal authority or your dealer for information about disposal.

Within the European Union, the marked equipment is covered by "Directive 2002/96/EC of the European Parliament and of the Council of 27 January 2003 on waste electrical and electronic equipment." Amended by "Directive 2003/108/EC" (WEEE).

HOW IT WORKS


The RIC (Receiver in Canal) device is a type of hearing aid that can be partially adapted to the EEC (External Ear Canal). It is similar to a behind-the-ear hearing aid, but in this case the receiver goes inside the EEC, fitted via a standard dome / adapter or a custom made earmould.

MICROPHONE: receives the sound and converts it into an electrical signal.

AMPLIFIER: digital processor that receives the signal and adapts it to the parameters introduced for each loss.

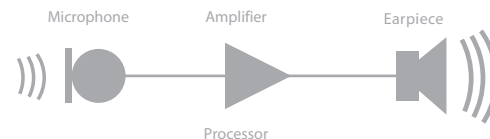RECEIVER: converts the electrical signal back into sound and transmits it into the user's ear.

 WARNING!

- It is dangerous to swallow batteries. Keep batteries out of the reach of children.
- Never use your hearing aids in places where there is medical equipment (X-ray, high frequency, magnetic resonance or short-wave therapy treatments).
- Do not use the hearing aids in areas where there is a risk of explosion!
- **m4 RIC / m2 RIC** hearing aids can generate output levels in excess of 132 dB_{SPL}.

 CAUTION!

- Protect your hearing aids against moisture and sources of heat.
- Do not use your hearing aids while showering, swimming or spraying your hair.
- Protect your hearing aids from hard knocks.
- Never use a microwave or other heat emitters to dry your hearing aids.
- Regular use of a dehumidifier is recommended to keep your hearing aid in optimum working conditions: Ask your hearing care specialist.

 INFORMATION

- Remove the batteries if you do not intend to use your hearing aids for a long time.
- Replace spent batteries immediately.
- Do not dispose of spent batteries with normal household waste! Take them to the hearing care specialist for proper disposal without harming the environment.

DOMES / ADAPTERS AND MOULDS WARNING!

Risk of injuries caused by incorrect use of the domes/adapters or moulds..

Always use the receiver tube with an receiver dome/adapter or mould.

Make sure that the receiver domes/adapters or moulds are correctly attached to the earpiece tube (press until you hear a click).

If the receiver dome / adapter or mould becomes lodged in the ear, ask a medical professional to remove it.

Carefully insert the earpiece tube in the ear with the earpiece dome/adapter or mould without inserting it too far.

Only use the earpiece tube and the earpiece dome/ adapter or mould as described in this User Manual.

Indicates situations where you may be injured.
Indicates unsafe situations or procedures that may lead to injury or damage to your hearing aid.
Indicates possible damage or alteration of the hearing aid or any of its components.

Operating instructions.
Indicates that it is important to follow the advice and warnings contained in this document.

Thanks to CE marking, Microson confirms that its hearing aids comply with European Directive 93/42 EEC concerning medical devices.

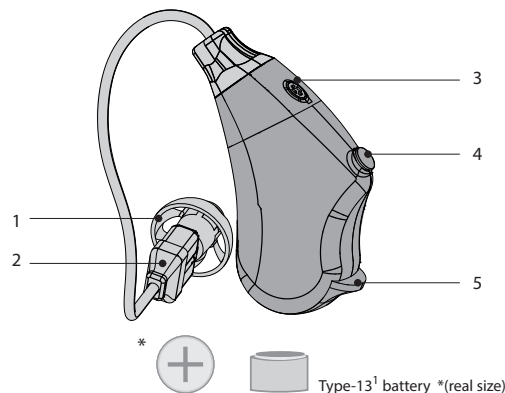
This symbol indicates that the products described in these user instructions adhere to the requirements for a Type B component under IEC 60601-2-66. The surface of the hearing aid is specified as a Type B component.

The symbol with the crossed-out garbage bin warns you that this hearing aid may not be disposed of with normal household waste. Please return your hearing aid to your hearing healthcare professional or a collection point for safe, environmentally-friendly disposal.

M4 RIC / M2 RIC

1. Dome or adaptor
2. Receiver
Right side --> Red colour
Left side --> Blue colour
3. Microphone
4. Programme button
5. Battery compartment

TYPES OF PERSONALISED DOMES / ADAPTERS AND MOULDS

Single standard dome or adapter

Double standard dome or adapter

Custom made ear mould

INFORMATION

See Options (page 17)

¹IEC 60086 Designation: PR48

VOLUME ADJUSTMENT

Your hearing aids are totally automatic, meaning that the volume adjusts automatically to the different acoustic environments where you happen to be.

You can also choose preset programmes in your hearing aids to adapt to your usual environments.

SWITCHING OFF VIA THE BATTERY COMPARTMENT

SWITCH OFF

SWITCH OFF:

To switch off your hearing aid, partially open the battery compartment.

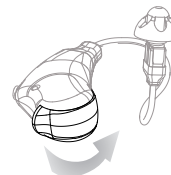
SWITCH ON

SWITCH ON:

To switch on your hearing aid, close the battery compartment.



BATTERY (INSERTION AND REPLACEMENT)

1. Open the battery compartment.


2. Insert the battery with the "+" symbol facing upwards and close the battery compartment.

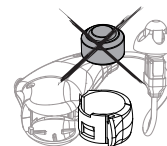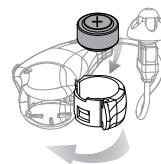 If the battery is not inserted correctly, the hearing aid will not work.

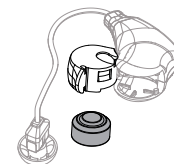
3. To remove the battery, turn the hearing aid upside down, open the battery compartment and allow it to drop into your hand.


REMEMBER THAT YOUR HEARING AID WILL BEEP FOUR TIMES  TO LET YOU KNOW WHEN THE BATTERY IS SPENT.

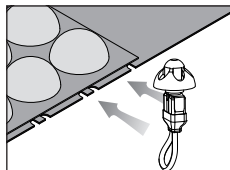
REPLACING THE ADAPTER

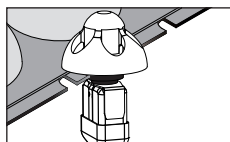
Fit the adapter into the notch of the dome package.

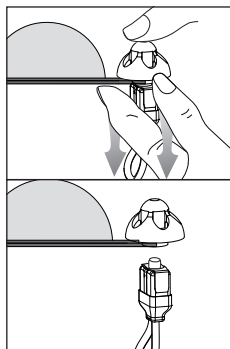Align the earpiece joint with the notch.

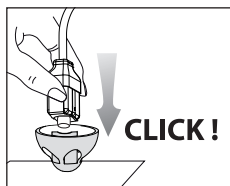
Hold the adapter and pull the receiver downwards to release it.

Insert the new dome in the receiver until you hear a click.

CLICK!

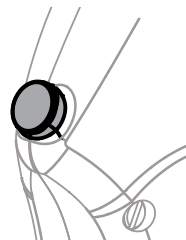PROGRAMME BUTTON


When you press the button you will hear tones that will let you know which programme you are using.

Example: (Two tones: programme 2)

Follow your hearing care specialist instructions to get the most from your hearing aids.

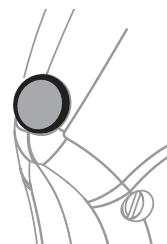PROGRAMME SELECTION

Each programme enables the device to be adapted to different environments or situations of everyday life.

For example, your hearing aids may have these programmes:

- 1- Normal.
- 2- Restaurant.
- 3- Telephone*.
- 4- Traffic.

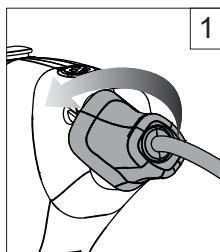
*Your hearing aids comply with the IEC 60118-13 electromagnetic compatibility standard.

Your hearing care specialist will change the receiver when necessary.


If you need to replace the receiver without assistance, follow these instructions carefully.

REMOVAL

1. Turn the receiver tube 45° counter clockwise.

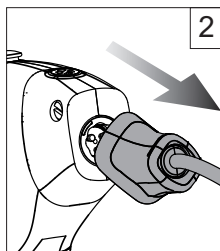
2. Remove the receiver tube.



INSERTION

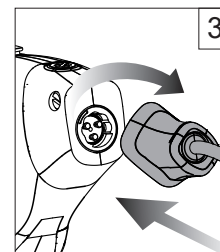3. Connect the hearing aid and the receiver tube.

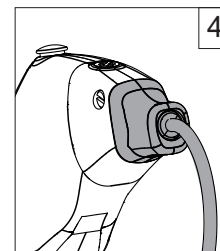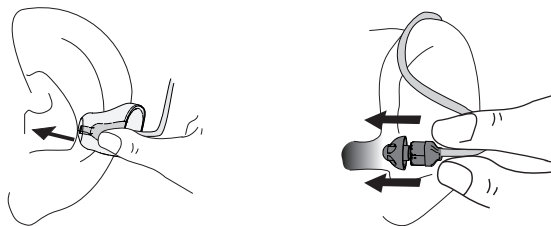
4. Turn the receiver tube 45° clockwise.

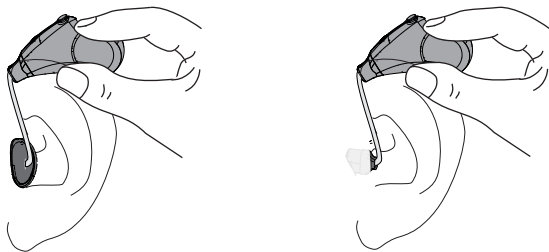Insert the mould or adapter into the ear canal.

Place the hearing aid behind the ear.

Check that the device is correctly supported.

INFORMATION

- Clean your hearing aid regularly, using specially designed wipes. (Consult your specialist).
- Regular use of a dehumidifier is recommended to keep your hearing aid in optimum working conditions. (Consult your specialist).
- Always have a spare battery available.

Do not wash your hearing aid!

DOMES / ADAPTERS

Cerumen (ear wax) in the domes / adapters or moulds affects sound quality.

Clean the domes / adapters or moulds immediately after removing the receiver with a soft cloth. No other tools are required.

ENVIRONMENTAL OPERATING CONDITIONS

For optimum operation we recommend using your hearing aids within a range of 0 / 40 °C (degrees Celsius). Likewise, the relative humidity during storage should not exceed 95%.

SYMPTOM	POSSIBLE CAUSE	SOLUTION
The hearing aid is not working	Spent battery Defective hearing aid	Replace battery Consult your hearing care specialist
The volume is too low	Dome / adapter or mould blocked or obstructed	Remove the obstruction and clean the dome / adapter or mould
Sound distorted or unclear	Spent battery	Replace battery

Manufacturer's
customer service:

+34 93 300 58 00
helpdesk@microson.es

NOTE

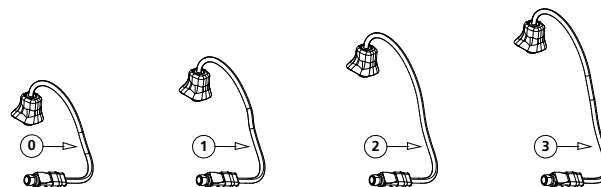
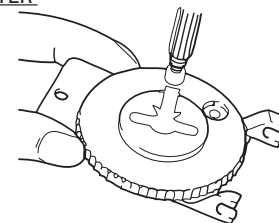
For any other problems or doubts, contact your audioprosthologist or retailer.

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