microson

microson

M4 BTE

M4 BTE AT

M4 BTE ECO

MU-0016-003 -LP2 RevE 2019-02-06

Printed in Spain

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ES

USER MANUAL

EN

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IT

MANUAL DO UTILIZADOR

PT

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FR

 MICROSON S.A.
Pere IV, 160
08005 BARCELONA, SPAIN
Tel. +34 93 300 58 00
www.microson.es

CE
0197

Ref. 91869

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Congratulations, welcome!

You have acquired a MICROSON hearing aid enhanced by the latest digital technology.

Your hearing aid is the result of years of research and development, designed for you to enjoy the sounds of life.

The Microson team is happy for your trust and wishes you much success in its usage.

 It is important that you read and follow the instructions in this manual to prevent damage to your hearing aid.

This manual has been produced in accordance with standards EN 1041 and EN ISO 15223-1.

Intended purpose	16
How it works	17
Explanation of symbols	18
Safety Instructions	19
Description of your hearing aid	20
Battery (insertion and replacement)	21
Volume Adjustment	22
Switching ON / OFF	22
Programs push button	23
Programs selection	23
Auto-Phone	24
Automatic Activation	24
Magnets (Safety warnings)	25
Maintenance and care of your Hearing Aid	26
Environmental conditions	26
Troubleshooting	27

INTENDED PURPOSE

The intended purpose of the hearing aids is to selectively amplify sound and transmit it to the ear to compensate for the hearing of hearing impaired people.

Its use is not intended for children or people with mental disabilities.


⚠ WARNING!

The prescription and adaptation of the hearing aids must be performed by a hearing health professional, who will perform the programming and specific adjustment for hearing loss and the morphology of the user in a completely personalized manner. Therefore, using hearing aids not prescribed by a professional or adapted for another person can damage your hearing.

⚠ WARNING!

Any product modification must be carried out by a manufacturer authorised Technical Service.

HOW IT WORKS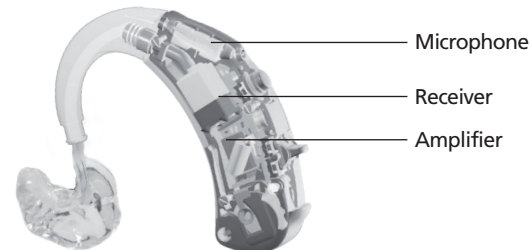


The BTE (Behind the Ear BTE) hearing aids are anatomical instruments placed behind the ear. Are powered by a battery and are typically designed to cover severe or profound hearing losses. All mechanical and electronic components are contained in an ergonomic housing. They adapt to the user's ear by molds which leaves a conductive tube that attaches to the side of the hearing aid.

MICROPHONE: receives the sound and converts it into an electrical signal.

AMPLIFIER: digital processor that receives the signal and adapts it to the parameters introduced for each loss.

RECEIVER: converts the electrical signal back into sound and transmits it into the user's ear.

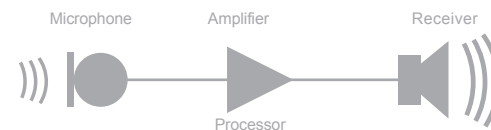EXPLANATION OF SYMBOLS

This symbol indicates that it is important for the user to pay attention to situations or procedures that could cause harm, damage or alteration to the hearing aid or any of its components.

Consult instructions for use
Indicates the need for the user to consult the instructions for use.

With the CE symbol, Microson confirms that the hearing aids comply with the essential requirements of European Directive 93/42/EEC for health products.
XXXX Identification of Notified Body

This symbol indicates that the products described in these instructions for use adhere to the requirements established for a Type B application component of the IEC.60601-2-66. The hearing aid surface is listed as a Type B application component.

This symbol indicates that the product must be managed differently than household waste at the end of its life. You must hand it over to your hearing care professional or at a point for the selective collection of electronic devices to protect health and the environment. DIRECTIVE OF THE EUROPEAN PARLIAMENT AND OF THE COUNCIL on waste electrical and electronic equipment (WEEE)

Manufacturer
Manufacturing Date (year - month)

Catalogue Number

Serial Number

SAFETY NOTES

⚠ WARNING!

- It is dangerous to swallow batteries. Keep batteries out of the reach of children.
- Never use your hearing aids in places where there is medical equipment (X-ray, high frequency, magnetic resonance or short-wave therapy treatments).
- Do not use the hearing aids in areas where there is a risk of explosion!
- **M4 BTE / M4 BTE AT / M4 BTE ECO** hearing aids can generate output levels in excess of 132 dB_{SPL}.

⚠ CAUTION!


- Protect your hearing aids against moisture and sources of heat.
- Do not use your hearing aids while showering, swimming or spraying your hair.
- Protect your hearing aids from hard knocks.
- Never use a microwave or other heat emitters to dry your hearing aids.
- Regular use of a dehumidifier is recommended to keep your hearing aid in optimum working conditions: Ask your hearing care specialist.

📖 INFORMATION



- Remove the batteries if you do not intend to use your hearing aids for a long time.
- Replace spent batteries immediately.
- Do not dispose of spent batteries with normal household waste! Take them to the hearing care specialist for proper disposal without harming the environment.

DESCRIPTION OF YOUR HEARING AID

M4 BTE / M4 BTE AT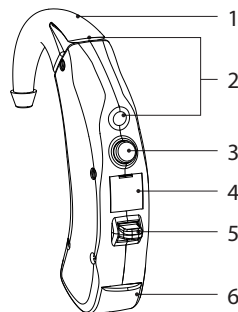

M4 BTE ECO

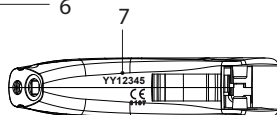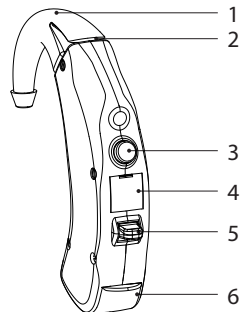
1. Ergonomic-ear-hook design
2. Microphone/s
3. Programs push button
4. Programming connector
(For use exclusively by the Hearing Aid specialist)
5. Volume Control
6. Battery compartment
7. Serial Number (YY: Year of Manufacturing)
8. Battery Type 13¹ *(Size original)

¹IEC 60086 Designation: PR48



BATTERY (INSERTION AND REPLACEMENT)

1. Open the battery compartment.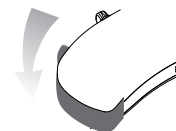


2. Insert the battery with the "+" symbol facing upwards and close the battery compartment.

 If the battery is not inserted correctly, the hearing aid will not work.

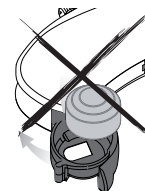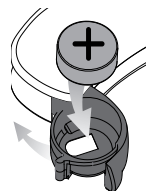
3. To remove the battery, turn the hearing aid upside down, open the battery compartment and allow it to drop into your hand.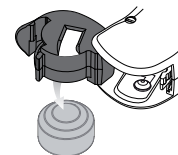

REMEMBER THAT YOUR HEARING AID WILL BEEP 5 TIMES  TO LET YOU KNOW WHEN THE BATTERY IS SPENT.


VOLUME ADJUSTMENT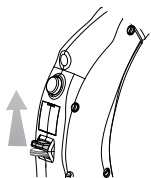

UP

Your hearing aid is totally automatic, but if you prefer manual adjustment, you can use the volume control.


Use your finger to operate the lever up or down, the volume of aircraft go up or down a fraction and a ring tone. Repeat this action until the volume desired.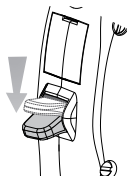

DOWN


If you reach the volume limit will warn you with two tones.

SWITCHING ON / OFF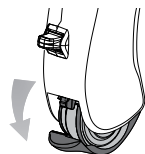

OFF


To disconnect your hearing aid, partially open the battery compartment.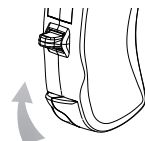

ON

To connect your hearing aid, close the battery compartment.

PROGRAMS PUSH BUTTON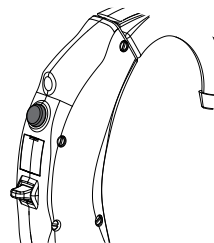


Your hearing aid has a push button that enables you to select the desired programme.

As you press the push button, you will hear tones indicating the programme you are using.

Example: Two tones ♪♪ for programme 2.

Follow the instructions of your audio prosthetist to get the most out of your hearing aid.

PROGRAMS SELECTION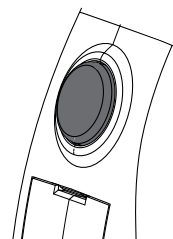


Each programme will permit you to better adapt to different environments or situations of everyday life.

By way of example, your hearing aid may be equipped with the following programmes:

- 1- Normal.
- 2- Restaurant.
- 3- Telephone*.
- 4- Traffic.

*Your hearing aid complies with the IEC 60118-13 standard on electromagnetic compatibility.


AUTO-PHONE (ONLY M4 BTE AT)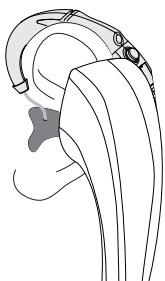

Your **M4 BTE AT** is equipped with the Auto-phone function that enables your hearing aid to select the telephone program automatically when you put the receiver to your ear.

When you finish the call and remove the phone from your ear, it will revert to the program you were using before.

AUTOMATIC ACTIVATION

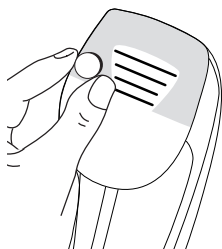To activate the Auto-phone automatically, place the magnetic accessory in your telephone's receiver. Move the magnet gently until it is in the optimum position* for a reliable connection and a pleasant sound. Next, remove the protector and place the accessory in the receiver.

MAGNETS (SAFETY WARNINGS)

⚠ CAUTION!

The magnets may affect certain electrical apparatuses. Be careful not to place the magnets near monitors, television sets or other electrical equipment.

Check that there are no credit cards, CDs or DVDs nearby, or other mechanisms sensitive to magnetic fields (minimum 30 cm).

⚠ WARNING!

- The magnets may interfere with the operation of medical equipment such as pacemakers and magnetic valves, etc.

Pacemaker users should follow the safety instructions of the pacemaker manufacturer and make use of the magnetic accessory when in no doubt about the safety distance. (Minimum 10 cm)

- Store the magnets out of children's reach. In the event a magnet is swallowed, consult your doctor.

MAINTENANCE AND CARE

INFORMATION

- Clean your hearing aid regularly, using specially designed wipes. (Consult your specialist).
- Regular use of a dehumidifier is recommended to keep your hearing aid in optimum working conditions. (Consult your specialist).
- Always have a spare battery available.

ATTENTION!

- Do not clean your hearing aid with alcohol, hydrogen peroxide or detergents: use a soft, moist cloth.
- Do not wash your hearing aid!

ENVIRONMENTAL CONDITIONS

		OPERATION		TRANSPORT AND STORAGE	
	Limits Temperature [T (°C)]	From 0 to 40 °C		From -20 to 60 °C	
	Relative Humidity [RH (%)]	From 10 to 95 % (without condensation)			
	Atmospheric pressure [P (hPa)]	500*	1100*	500*	1100*

*Avoid rapid pressure changes

TROUBLESHOOTING

SYMPTOM	POSSIBLE CAUSE	SOLUTION
The hearing aid does not work	Worn battery defective hearing aid	Replace battery consult your specialist
Insufficient volume	Mould blocked or obstructed	Eliminate obstruction and clean the mould
Sound distorted or unclear	Worn battery	Replace battery

Manufacturer's
customer service:

 +34 93 300 58 00
 helpdesk@microson.es

NOTE

For any other problems or doubts, contact your audioprosthologist or retailer.

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