microson

MU-0025-002-LP3 Rev.C 2014-10-02

Free ITC

Printed in Spain

USER'S MANUAL

EN

KULLANIM KILAVUZU

TR

MANUAL DE UTILIZARE

RO

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microson

Congratulations and welcome!

You have bought a MICROSON hearing aid featuring the latest digital technology.

Your new hearing aid is the result of years of research and development and is designed to make sure you enjoy all the sounds of life.

The Microson team would like to wish you great success with your hearing aid.

It is important that you read and follow the instructions in this manual to prevent injury or any damage to your hearing aid. This manual was drawn up in compliance with the EN 1041 and ISO 15223-1 standards.

Intended use 2

Information for disposal 2

Principles of operation 3

Explanation of symbols 4

Safety notes 5

Description of your ear 6

Position of the device in your ear 6

Description of your hearing aid 7

Fitting recommendations 8

Battery (insertion and replacement) 9

Inserting your hearing aid 10

Inserting the D-Ring 11

Changing the D-Ring 12

Removing the D-Ring 12

Removing your hearing aid 13

Changing programme 13

Care and maintenance 14

Environmental operating conditions 14

Troubleshooting 15

The intended purpose of the hearing aids is to amplify sound selectively and to transmit it to the ears of people with a hearing disability (the hard of hearing) to compensate for their hearing loss.

Use of the MICROSON Free ITC hearing aid is recommended to compensate for mild to moderate hearing loss (mixed or sensorineural). It is not recommended for children.

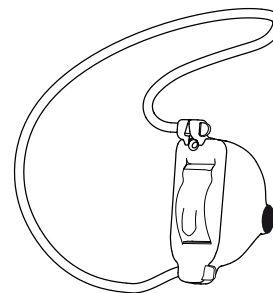
A hearing aid must be prescribed and adapted by a professional audiologist, who will programme it and adjust it specifically to the patient's hearing loss and morphology in a completely personalised way. Hence using hearing aids not prescribed by a professional or adapted for another person may damage your hearing.

INFORMATION FOR DISPOSAL

This product must not be disposed of with normal household waste. All users are obliged to dispose of all electrical or electronic devices, at a local or commercial collection point specialising in environmental conservation and protection. When your hearing aid is no longer in use, return it to your hearing healthcare professional or collection point for treatment and recycling.

Within the European Union, marked devices are governed by "Directive 2002/96/EC of the European Parliament and of the Council of 27 January 2003 on waste electrical and electronic equipment." Amended by "Directive 2003/108/EC" (WEEE).


This ITC (In-the-Canal) device is a hearing aid placed in the external ear canal (EEC) using a moulded shell containing all mechanical and electronic elements. It is powered by a battery, and is appropriate for correcting mild to moderate hearing loss.

Shell: part of the device fitted into the ear canal.

Microphone: receives the sound and converts it into an electrical signal.

Amplifier: digital processor that receives the signal and adapts it to the parameters entered for each type of hearing loss.

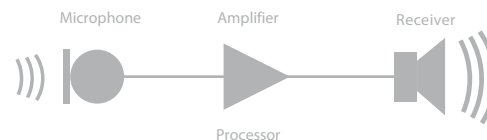Receiver: converts the electrical signal back into sound and transmits it into the user's ear.

This symbol indicates unsafe situations or procedures that may lead to injury or cause damage to your hearing aid or any of its components.

Operating instructions.

This symbol indicates that it is important to follow the advice and warnings contained in this document.

The CE Mark enables Microson to confirm that its hearing aids comply with the essential requirements with European Directive 93/42/EEC concerning medical devices.

This symbol indicates that the products described in these user instructions adhere to the requirements for a Type B component under IEC 60601-2-66. The surface of the hearing aid is specified as a Type B component.

The symbol with the crossed-out garbage bin warns you that this hearing aid may not be disposed of with normal household waste. Please return your hearing aid to your hearing healthcare professional or a collection point for safe, environmentally-friendly disposal.

WARNING!

- Batteries are dangerous if swallowed. Keep batteries out of the reach of children.
- Never use your hearing aids in places where there is medical equipment (X-ray, high frequency, magnetic resonance or short-wave therapy treatments).
- Do not use your hearing aid in areas where there is a risk of explosion!

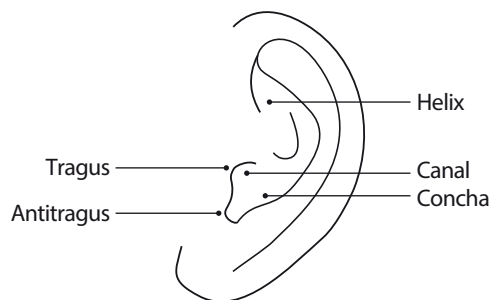
CAUTION!

- Protect your hearing aid against moisture and sources of heat.
- Do not use your hearing aid while showering, swimming or spraying your hair.
- Protect your hearing aid from hard knocks.
- Never use a microwave or other heating device to dry your hearing aid.

INFORMATION


- Remove the batteries if you do not intend to use your hearing aids for a long period.
- Replace spent batteries immediately.
- Do not dispose of spent batteries with normal household waste! Take them to your hearing healthcare professional for proper disposal without harming the environment.

DESCRIPTION OF YOUR EAR


POSITION OF THE DEVICE IN YOUR EAR

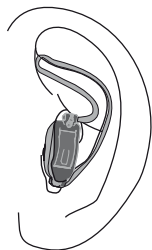The device will ideally sit slightly inserted between the tragus and the canal wall. When inserted correctly, the hearing aid should not be visible from the front and only partially visible from the side. The D-Ring should be inserted well into the concha and helix to ensure the aid is properly secured.

DESCRIPTION OF YOUR HEARING AID

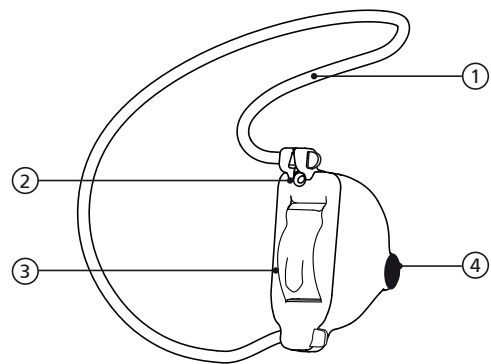Free ITC

Battery Size 10¹ *Actual size

- ① D-Ring retention system
- ② Microphone
- ③ Battery compartment (Size10 - PR70)
- ④ Wax filter

¹IEC 60086 Designation: PR70 Battery

FITTING RECOMMENDATIONS

Your hearing healthcare professional must ensure your ear canal is long enough to hold the Free ITC properly. (See Inserting your Hearing Aid). If your ear canal is too small, Free ITC is not an appropriate hearing aid for you.

Connect the Medium D-Ring system to the Free ITC (Connecting the D-Ring to the Hearing Aid) and insert the Free ITC into the ear (Inserting your Hearing Aid).

⚠ If the D-Ring does not fit snugly, or if it is too tight or becomes deformed, you will need to try another size.

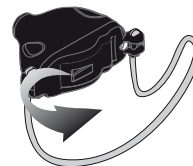
⚠ Check that the D-Ring is not too loose, as this may prevent the device from being held in place. (A slightly looser fit is preferable to one that is too tight).

⚠ Your Free ITC with a D-Ring should make contact with several points of the ear (around the concha wall and fold of the helix).


If you cannot find a size that fits correctly, you may wish to consider a hearing aid with a custom-made mould/shell.

BATTERY (INSERTION AND REPLACEMENT)


1- Open the battery compartment.

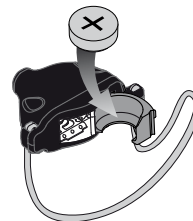2- Insert the battery and close the battery compartment.

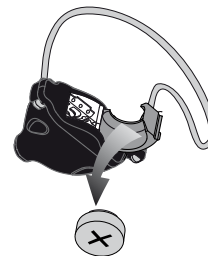
3- To remove the battery, turn the hearing aid upside down and let it drop into your hand.

INSERTING YOUR HEARING AID

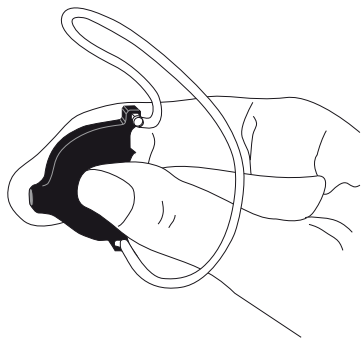To insert your hearing aid, hold it between your thumb and index finger and bring it up to your ear.


Place the tip of the hearing aid into the ear canal, towards the ear drum, and slide it gently into place.


INSERTING THE D-RING

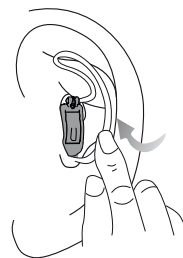Next place the D-Ring in position around the concha, from the bottom upwards. Make sure that the top of the D-Ring is tucked well under the fold of the helix.

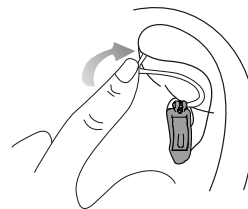
Run your finger along the adaptor wire in a D-shape to ensure that the D-Ring does not slide out of the concha.

CHANGING THE D-RING

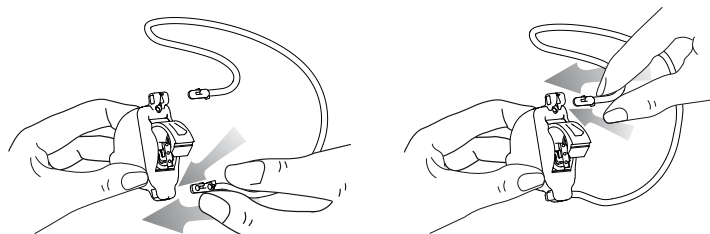Check that the colour (indicating the side) matches that of the battery compartment and that the D-Ring is the correct size. Hold the hearing aid in one hand and the D-Ring in the other. First connect the bottom end of the D-Ring by pushing it into the bottom clip of the hearing aid. Then connect the top of the D-Ring by pushing it into the top clip.


REMOVING THE D-RING

Hold the hearing aid in one hand and the D-Ring in the other. Pull the upper and lower clips with the other hand (in the opposite direction to the arrows in the illustration above) until they come away from the hearing aid. It is important to hold the D-Ring as close to the clips as possible when pulling to prevent causing any damage.

REMOVING YOUR HEARING AID

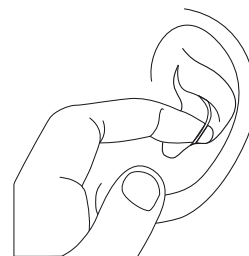Run your fingertip around the edge of the concha until you find the D-Ring tube. Lift it away from the concha and gently pull to remove the hearing aid from your ear.

CHANGING PROGRAMME

Your hearing aid is fitted with a noise-sensitive programme change. To activate this function, bring your hand up (with your fingers together) parallel to your ear until you hear the sound confirming the programme change. This may require some practice. Your hearing healthcare professional can help you to use and make the most of this feature.

 INFORMATION

- Clean your hearing aid regularly, using specially designed wipes. (Consult your specialist).
- Regular use of a dehumidifier is recommended to keep your hearing aid in optimum working conditions. (Consult your specialist).
- Always have a spare battery available.

 ATTENTION!

- Do not clean your hearing aid with alcohol, hydrogen peroxide or detergents: use a soft, moist cloth.

 Do not wash your hearing aid!

ENVIRONMENTAL OPERATING CONDITIONS

For optimum operation we recommend using your hearing aids within a temperature range of 0 / 40 °C (degrees Celsius). Likewise, the relative humidity during storage should not exceed 95%.

SYMPTOM	POSSIBLE CAUSE	SOLUTION
The hearing aid does not work	Spent battery	Replace battery
	Defective hearing aid	Consult your specialist
Volume too low	Blocked or obstructed filter	Remove obstruction
		Replace filter
Distorted or unclear sound	Spent battery	Replace battery

Manufacturer's customer service:  +34 93 300 58 00
 helpdesk@microson.es

 NOTE

For any other problems or queries, please contact your hearing healthcare professional or retailer.

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