microson

MU-0042-001-LP2 RevA 2015-06-12

m2

m2 ITC P

m2 ITC

m2 CIC

Printed in Spain

MANUAL DE USO

ES

USER'S MANUAL

EN

MANUALE D'USO

IT

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PT

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FR

REF 94116

MICROSON S.A.
Pere IV, 160
08005 BARCELONA, SPAIN
Tel. +34 93 300 58 00
www.microson.es

microson

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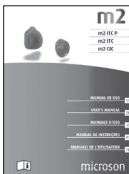
Congratulations and welcome!

You have bought a MICROSON hearing aid featuring the latest digital technology.

Your new hearing aid is the result of years of research and development and is designed to make sure you enjoy all the sounds of life.

The Microson team would like to wish you great success with your fitting.

It is important that you read and follow the instructions in this manual to prevent injury or any damage to your hearing aid.

This manual has been drawn up in compliance with the EN 1041 and EN ISO 15223-1 standards.


Intended use	14
How it works	15
Explanation of symbols	16
Safety notes	17
Description of your hearing aid	18
Battery (insertion and replacement)	19
Switching off via the battery compartment	19
Inserting the instrument	20
Remove the instrument	20
Programs push button	21
Programs selection	21
Care and maintenance of your hearing aid	22
Environmental conditions	22
Troubleshooting	23
Information for disposal	24
Regulatory compliance	24
Accessories and spare parts	25

The intended purpose of the hearing aids is to amplify sound selectively and to transmit it to the ears of people with a hearing disability (the hard of hearing) to compensate for their hearing loss.

The MICROSON m2 ITC P is indicated to compensate moderate to severe hearing loss (conductive, mixed or neurosensorial). Its use in children is not recommended.

The MICROSON m2 ITC and MICROSON m2 CIC are indicated to compensate mild to moderate hearing loss (conductive, mixed or neurosensorial). Its use in children is not recommended.

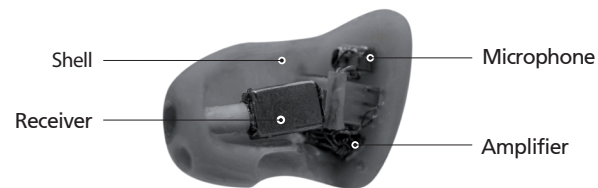A hearing aid must be prescribed and fitted by a hearing care specialist, who will program it and adjust it specifically for the patient's hearing loss and morphology in a completely personalised way. Be aware that using hearing aids not prescribed by a professional or devices fitted for other people can damage your hearing.


This ITC (In-the-Canal) device is a hearing aid placed in the external ear canal (EEC) using a moulded shell containing all mechanical and electronic elements. It is powered by a battery, and is appropriate for correcting mild to moderate hearing loss .

Shell: Part of the device fitted into the ear canal.

Microphone: Receives the sound and converts it into an electrical signal.

Amplifier: Digital processor that receives the signal and adapts it to the parameters entered for each type of hearing loss.

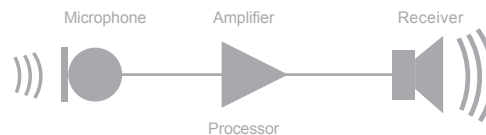Receiver: Converts the electrical signal back into sound and transmits it into the user's ear.

This symbol indicates unsafe situations or procedures that may lead to injury or cause damage to your hearing aid or any of its components.

Operating instructions.

This symbol indicates that it is important to follow the advice and warnings contained in this document.

The CE Mark enables Microson to confirm that its hearing aids comply with European Union Council Directive 93/42/EEC concerning medical devices.

This symbol indicates that the products described in this User Manual adhere to the requirements for a Type B component under IEC 60601-2-66. The surface of the hearing aid is specified as a B component.

The symbol warns you that this hearing aid may not be disposed of with normal household waste. Please return your hearing aid to your hearing care specialist or to a collection point for safe, environmentally-friendly disposal.

⚠ WARNING!

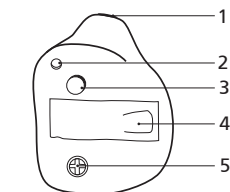
- Batteries are dangerous if swallowed. Keep batteries out of the reach of children.
- Never use your hearing aids in places where there is medical equipment (X-ray, high frequency, magnetic resonance or short-wave therapy treatments).
- Do not use your hearing aids in areas where there is a risk of explosion!

⚠ CAUTION!

- Protect your hearing aids against moisture and sources of heat.
- Do not use your hearing aids while showering, swimming or spraying your hair.
- Protect your hearing aids from hard knocks.
- Never use a microwave or other heating devices to dry your hearing aids.
- Regular use of a dehumidifier is recommended to keep your hearing aids in optimum working conditions. Ask your hearing care specialist.

📖 INFORMATION


- Remove the batteries if you do not intend to use your hearing aids for a long period.
- Replace spent batteries immediately.
- Do not dispose of spent batteries with normal household waste! Take them to your hearing care specialist for proper disposal without harming the environment.

Battery Size 312¹ * (Size original)

1. Wax Filter
2. Ventilation
3. Programmes push button
4. Battery compartment
5. Microphone

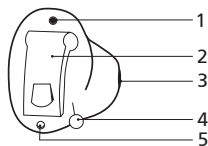
¹IEC 60086 Designation: PR41


Battery Size 10² * (Size original)

1. Microphone
2. Battery compartment
3. Wax Filter
4. Extractor
5. Ventilation

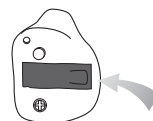¹IEC 60086 Designation: PR70

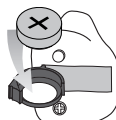
1. Open the battery compartment.

2. Insert the battery with the + symbol facing upwards and close the battery compartment.

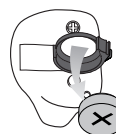 If the battery is not inserted correctly, the hearing aid will not work.

3. To remove the battery, turn the hearing aid upside down, open the battery compartment and drop it into your hand.

REMEMBER THAT YOUR HEARING AID GIVES A WARNING OF 2 SOUND SIGNALS 🎵 TO INFORM YOU THAT THE BATTERY IS DEAD.

SWITCH OFF

SWITCH OFF:

To switch off your hearing aids, partially open the battery compartment.

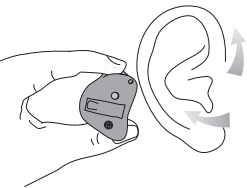
SWITCH ON


SWITCH ON:

To switch on your hearing aid, close the battery compartment.

INSERTING THE INSTRUMENT

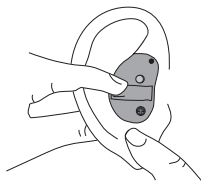1. Hold your hearing aid with your thumb and your index finger and place it carefully in the ear canal. Move the outer ear slightly upwards or downwards to help.


2. Insert the hearing aid completely by pushing very gently. The hearing aid must feel to be properly in place.

REMOVE THE INSTRUMENT


3. To take your hearing aid out, pull the ear lobe to loosen the hearing aid in the ear canal. Use the other hand to open the battery holder and pull gently on the hearing aid to remove it.

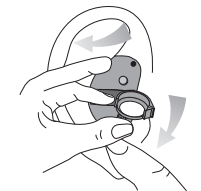NOTE: If your device has an extractor, take hold of the end and pull the hearing aid slowly until it has been removed.


PROGRAMS PUSH BUTTON

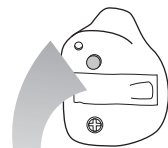Your hearing aid has a push button that enables you to select the desired program.

As you press the push button, you will hear tones indicating the programme you are using.

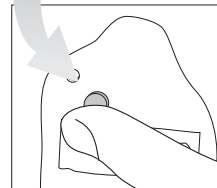Example: Two tones ♪♪ for program 2.


Follow the instructions of your audio prosthetist to get the most out of your hearing aid.

PROGRAMS SELECTION

Each program will permit you to better adapt to different environments or situations of everyday life.

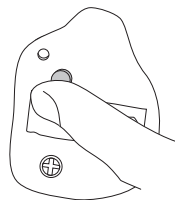By way of example, your hearing aid may be equipped with the following programs:

- 1- Normal.
- 2- Restaurant.
- 3- Telephone*.
- 4- Traffic.

*Your hearing aid complies with the IEC 60118-13 standard on electromagnetic compatibility.

 INFORMATION

- Clean your hearing instruments regularly, using specially designed wipe. (Ask your hearing care specialist).
- Regular use of a dehumidifier is recommended to keep your hearing aids in optimum working conditions. (Ask your hearing care specialist).
- Always have a spare battery available.

 ATTENTION!

- Do not clean your hearing aids with alcohol, hydrogen peroxide or detergents: use a soft, moist cloth.

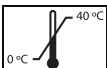
Do not wash your hearing aids! Keep them dry and protected from humidity or damp.

ENVIRONMENTAL CONDITIONS

RECOMMENDED USE AND STORAGE:

For optimum operation we recommend using your hearing aids within a temperature range of 0/40 °C (dregees Celsius). Likewise, the relative humidity during storage should not exceed 95% (non-condensing).

SYMPTOM	POSSIBLE CAUSE	SOLUTION
The hearing aid does not work	Spent battery	Replace battery
	Defective hearing aid	Consult your hearing care specialist
Volume too low	Blocked or dirty waxfilter	Remove the obstruction and clean the waxfilter
Sound distorted or unclear	Spent battery	Replace battery

Manufacturer's customer service:  +34 93 300 58 00  helpdesk@microson.es

 NOTE

For any other problems or queries, please contact your hearing care specialist or retailer.

INFORMATION FOR DISPOSAL

This product must not be disposed of with normal household waste. All users are obliged to dispose of all electrical or electronic devices at a local or commercial collection point specialising in environmental conservation and protection. When your hearing aid is no longer in use, return it to your hearing care specialist or collection point for treatment and recycling.

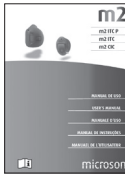
Within the European Union, marked devices are governed by Directive 2002/96/EC of the European Parliament and of the Council on waste electrical and electronic equipment (WEE).

REGULATORY COMPLIANCE

The CE Mark enables Microson to confirm that its hearing aids comply with European Directive 93/42/EEC concerning medical devices.

ACCESSORIES AND SPARE PARTS

63850	Blue HF3 pack filters + dispenser (15 pcs).	
63849	Red HF3 pack filters + dispenser (15 pcs).	
82879	m2 ITC P/ m2 ITC 6 pack of Microson 312 hearing aid batteries.	
82878	m2 CIC 6 pack of Microson 10 hearing aid batteries.	
91118	Brush cleaner	
88192	Microson Microbox case.	
94116	m2 user manual. m2 ITC P/ m2 ITC / m2 CIC (LP2)	

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