microson

microson


MU-0042-004-LP2 Rev.A 2019-04-09

m4 ITC P ZERO

m4 CIC P ZERO

m2 ITC ZERO

m2 CIC ZERO

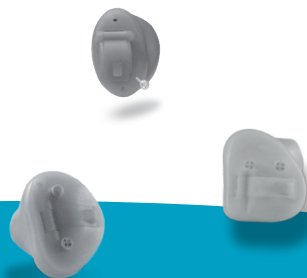Printed in Spain

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ES

USER MANUAL

EN

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IT

MANUAL DO UTILIZADOR

PT

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FR

Ref. 105233

MICROSON S.A.U.
Pere IV, 160
08005 BARCELONA, SPAIN
Tel. +34 93 300 58 00
www.microson.es

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Congratulations, welcome!

You have acquired a MICROSON hearing aid enhanced by the latest digital technology.

Your hearing aid is the result of years of research and development, designed for you to enjoy the sounds of life.

The Microson team is happy for your trust and wishes you much success in its usage.

It is important that you read and follow the instructions in this manual to prevent damage to your hearing aid.

This manual has been produced in accordance with standards EN 1041 and EN ISO 15223-1.

This manual can be downloaded from the Microson website **www.microson.es**.

Description of symbols_____ 14

Operation principles_____ 15

Ambient conditions_____ 15

Intended use_____ 16

Safety instructions_____ 17

Description of hearing aid_____ 18

Battery (insertion and change)_____ 19

Disconnection of the hearing aid _____ 19

Inserting the instrument_____ 20

Remove the instrument_____ 20

Program Push Button_____ 21

Program selection_____ 21

Maintenance and care_____ 22

Anti-earwax filter replacement_____ 22

Solution to possible problems_____ 23

Accessories and spare parts_____ 23

DESCRIPTION OF SYMBOLS

This symbol indicates that it is important for the user to pay attention to situations or procedures that could cause harm, damage or alteration to the hearing aid or any of its components.

Instructions of operation/Consult the instructions of use
Indicates the need for the user to consult the instructions for use and follow the advice and warnings contained in this document.

With the CE symbol, Microson confirms that the hearing aids comply with the essential requirements of European Directive 93/42/EEC for health products.
XXXX Identification of Notified Body

This symbol indicates that the products described in these instructions for use adhere to the requirements established for a Type B application component of the IEC.60601-2-66. The hearing aid surface is listed as a Type B application component.

This symbol indicates that the product must be managed differently than household waste at the end of its life. You must hand it over to your hearing care professional or at a point for the selective collection of electronic devices to protect health and the environment.

Manufacturer
Manufacturing Date (year - month)


Catalogue Number

Serial Number

OPERATION PRINCIPLES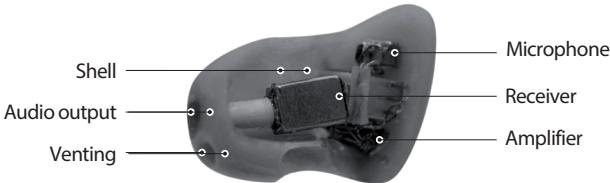

The Canal (ITC In-the-Canal) is a hearing aid that fits in the ear canal (EAM) using a custom designed shell, where all the mechanic and electronic parts are inserted into it. It is powered by a battery and is intended to be fitted from mild to moderate hearing losses.

MICROPHONE: receives the sound and converts it into an electrical signal.

AMPLIFIER: digital processor that receives the signal and adapts it to the parameters introduced for each loss.

RECEIVER: converts the electrical signal back into sound and transmits it into the user's ear.

AMBIENT CONDITIONS

		OPERATION		TRANSPORT AND STORAGE	
	Limits Temperature [T (°C)]	From 0 to 40 °C		From -20 to 60 °C	
	Relative Humidity [RH (%)]	From 10 to 95 % (without condensation)			
	Atmospheric Pressure [P (hPa)]	500*	1100*	500*	1100*

*Avoid rapid pressure changes

The intended purpose of the hearing aids is to selectively amplify sound and transmit it to the ear to compensate for the hearing of hearing impaired people.

The hearing aids are intended to compensate for mild to severe (mixed or neurosensory) hearing loss. Its use is not intended for children or people with mental disabilities.

 WARNING!

The prescription and adaptation of the hearing aids must be performed by a hearing health professional, who will perform the programming and specific adjustment for hearing loss and the morphology of the user in a completely personalized manner. Therefore, using hearing aids not prescribed by a professional or adapted for another person can damage your hearing.

 Any modification of the product must be carried out by a Technical Service authorised by the manufacturer.


	MILD	MODERATE	SEVERE	PROFOUND
m4 ITC P DIR ZERO				
m4 CIC P ZERO				
m2 ITC ZERO				
m2 CIC ZERO				

- Swallowing the batteries is harmful. Keep out of reach of children or people with mental disabilities.
- Consult before using your hearing aids in zones in which electronic or wireless devices are restricted (aeroplanes, for example).
- Do not under any circumstances use your hearing aids in areas with medical equipment (X-ray, high-frequency, MRI or shortwave therapy).
- Do not use the hearing aids in areas exposed to the danger of explosion or an oxygen-rich atmosphere!
- Stay away from strong electromagnetic fields.
- Persons with pacemakers or magnetic valves should keep their hearing aids at least 15 cm away; if interference is detected, refer to the implant manufacturer’s safety instructions. Metal detectors or alarms may cause interference or create unexpected sounds.
- The hearing aids can generate major output levels to 132 dB_{SPL}. A hearing health professional should take special care when adapting this hearing aid, as there may be a risk of damaging the user’s residual hearing.


 WARNING!

- Protect your hearing aids from moisture, heat sources.
- Do not use your hearing aids if you are going to shower, bathe or spray your hair.
- Do not let your hearing aids to suffer hard impacts.
- Never use a microwave or other heat-emitting devices to dry your hearing aids.


m4 ITC P DIR ZERO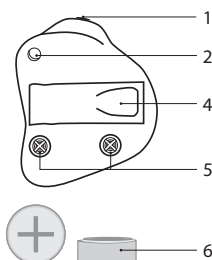


m2 ITC ZERO

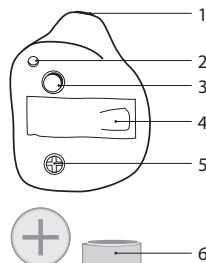m4 CIC P ZERO

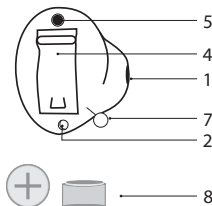m2 CIC ZERO

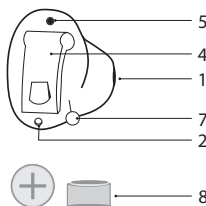1. Wax Filter
2. Venting
3. Button for changing programs*
4. Battery compartment
5. Microphone/s
6. Battery Size 312, IEC 60086: PR41 (Size original)
7. Extractor
8. Battery Size 10, IEC 60086: PR70 (Size original)


Side Indicator

Blue --> LEFT Side

Red --> RIGHT Side

*(Option for m4 ITC P DIR ZERO / m4 CIC P ZERO)


BATTERY (INSERTION AND CHANGE)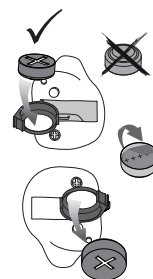

- 1- Open the battery compartment.
- 2- Remove the battery protector and place the new battery with the "+" symbol facing upwards and close the battery compartment. ⚠ If the battery is not inserted correctly, the hearing aid will not work.
- 3- To remove the battery, turn the instrument upside down and let the battery fall into your hand.


INFORMATION

- Remove the batteries if you will not use the hearing aid for a prolonged period.
 - Immediately replace weak batteries.
- ⚠ Do not throw the batteries in the garbage! Hand them in to a hearing health professional so that the batteries are properly eliminated without harming the environment.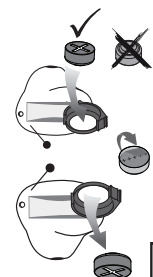

REMEMBER THAT YOUR HEARING AID GIVES A WARNING OF 4 SOUND SIGNALS TO INFORM YOU THAT THE BATTERY IS DEAD.

DISCONNECTION OF THE HEARING AID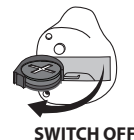

SWITCH OFF:


To switch off your hearing aids, partially open the battery compartment.

SWITCH ON:


To switch on your hearing aid, close the battery compartment.

INSERTING THE INSTRUMENT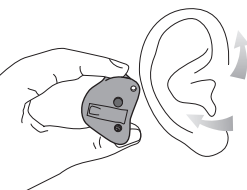


1. Hold your hearing aid with your thumb and your index finger and place it carefully in the ear canal. Move the outer ear slightly upwards or downwards to help.


2. Insert the hearing aid completely by pushing very gently. The hearing aid must feel to be properly in place.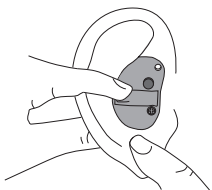


REMOVE THE INSTRUMENT

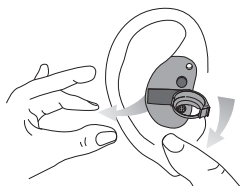3. To take your hearing aid out, pull the ear lobe to loosen the hearing aid in the ear canal. Use the other hand to open the battery holder and pull gently on the hearing aid to remove it. NOTE: If your device has an extractor, take hold of the end and pull the hearing aid slowly until it has been removed.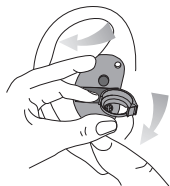

PROGRAM PUSH BUTTON (OPTION)

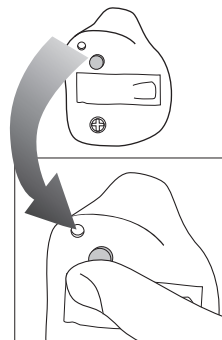Your hearing aid has a push button that enables you to select the desired program.

As you press the push button, you will hear tones indicating the programme you are using.


Example: Two tones 🎵 for program 2.

Follow the instructions of your audio prosthetist to get the most out of your hearing aid.

PROGRAM SELECTION

Each program will permit you to better adapt to different environments or situations of everyday life.

By way of example, your hearing aid may be equipped with the following programs: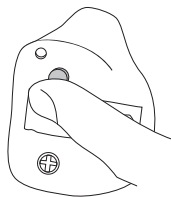


- 1- Normal.
- 2- Restaurant.
- 3- Telephone*.
- 4- Traffic.

*Your hearing aid complies with the IEC 60118-13 standard on electromagnetic compatibility.


MAINTENANCE AND CARE

DAILY
Cleaning the sound output.
Dehumidifier treatment. (Consult your hearing care professional)
External cleaning of hearing aid.
WEEKLY
Unclog the air vents of hearing aid.
Cleaning the battery compartment.


ANTI-EARWAX FILTER REPLACEMENT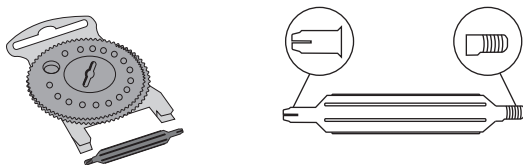

Anti-earwax filter removal

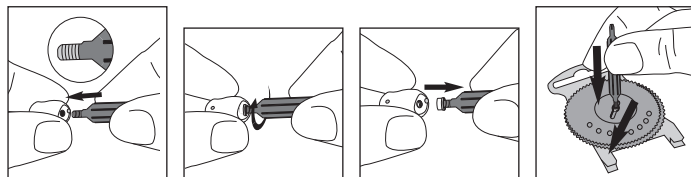Fitting a new anti-earwax filter

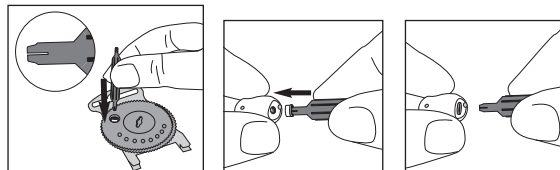⚠ WARNING!

If the anti-earwax filter detaches from the hearing aid and remains loose in the ear canal, please contact your doctor immediately. Do not try to remove the anti-earwax filter from the ear canal yourself.

Never reuse a used anti-earwax filter.

SOLUTION TO POSSIBLE PROBLEMS

SYMPTOM	POSSIBLE CAUSE	SOLUTION
The hearing aid does not work	Spent battery Defective hearing aid	Replace battery Consult your hearing care specialist
Insufficient volume	Blocked or dirty waxfilter	Remove the obstruction and clean the waxfilter
Sound distorted or unclear	Spent battery	Replace battery

Manufacturer's customer service:

☎ +34 93 300 58 00

NOTE

✉ helpdesk@microson.es

Contact your hearing care professional or distributor about any other problem or doubt.

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