microson

MU10057 Rev. H 2012-07-09

M500 duo

M500 duo ITC P

M500 duo ITC

M500 duo CIC

Printed in Spain

MANUAL DE USO

ES

USER'S MANUAL

EN

MANUALE D'USO

IT

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PT

KULLANIM KILAVUZU

TR

MANUAL DE UTILIZARE

RO

MICROSON S.A.
Pere IV, 160
08005 BARCELONA, SPAIN
Tel. +34 93 300 58 00
www.microson.es

Ref. 66169

microson

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Congratulations and welcome!

You have bought a MICROSON hearing aid featuring the latest digital technology.

Your hearing device is the result of years of research and development and is designed to make sure you enjoy all the sounds of life.

The team at Microson would like to wish you great success with it.

Intended use	14
Disposal Information	14
How it works	15
Explanation of symbols	16
Safety Instructions	17
Description of your hearing aid	18
Battery (insertion and replacement)	19
Inserting the instrument	20
Remove the instrument	20
Programs push button	21
Programs selection	21
Care and maintenance of your hearing aid	22
Environmental operating conditions	22
Troubleshooting	23

INTENDED USE

Microson hearing aids are designed for correcting hearing problems and compensating for hearing loss by selective sound amplification. Your hearing aids should be fitted by a hearing care specialist.

For safe use, read the instructions in this manual carefully.

This manual was drawn up in compliance with the EN 1041 and EN 980 standards.


DISPOSAL INFORMATION

This product must not be disposed together with the domestic waste. All users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner. Remove the batteries before disposing of the equipment. Do not dispose of old batteries with your household waste, but dispose of them at a battery collection station at a recycling site or in a shop. Consult your municipal authority or your dealer for information about disposal.

Within the European Union, the marked equipment is covered by "Directive 2002/96/EC of the European Parliament and of the Council of 27 January 2003 on waste electrical and electronic equipment." Amended by "Directive 2003/108/EC" (WEEE).

HOW IT WORKS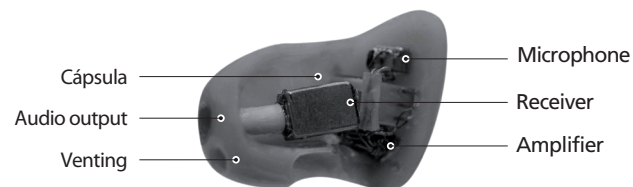


The Canal (ITC In-the-Canal) is a hearing aid that fits in the ear canal (EAM) using a custom designed shell, where all the mechanic and electronic parts are inserted into it. It is powered by a battery and is intended to be fitted from mild to moderate hearing losses.

MICROPHONE: receives the sound and converts it into an electrical signal.

AMPLIFIER: digital processor that receives the signal and adapts it to the parameters introduced for each loss.

RECEIVER: converts the electrical signal back into sound and transmits it into the user's ear.

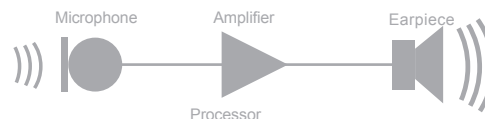Indicates situations where you may be injured.

Indicates unsafe situations or procedures that may lead to injury or damage to your hearing aid.

Indicates possible damage or alteration of the hearing aid or any of its components.

Operating instructions.

Indicates that it is important to follow the advice and warnings contained in this document.

Thanks to CE marking, Microson confirms that its hearing aids comply with European Directive 93/42 EEC concerning medical devices.

This symbol indicates that the products described in these user instructions adhere to the requirements for an application part of Type BF of EN 60601-1. The surface of the hearing aid is specified as application part of Type BF.

The symbol with the crossed-out garbage bin is to make you aware that this hearing aid may not be thrown away as normal household waste. Please dispose of old or unused hearing aids, at waste disposal sites intended for electronic waste, or give your hearing aid to your hearing care professional for disposal. Proper disposal protects the environment and health.

⚠ WARNING!

- It is dangerous to swallow batteries. Keep batteries out of the reach of children.
- Never use your hearing aids in places where there is medical equipment (X-ray, high frequency, magnetic resonance or short-wave therapy treatments).
- Do not use the hearing aids in areas where there is a risk of explosion!
- **M500 duo** hearing aids can generate output levels in excess of 132 dB_{SPL}.


⚠ CAUTION!

- Protect your hearing aids against moisture and sources of heat.
- Do not use your hearing aids while showering, swimming or spraying your hair.
- Protect your hearing aids from hard knocks.
- Never use a microwave or other heat emitters to dry your hearing aids.
- Regular use of a dehumidifier is recommended to keep your hearing aid in optimum working conditions: Ask your hearing care specialist.

📖 INFORMATION

- Remove the batteries if you do not intend to use your hearing aids for a long time.
- Replace spent batteries immediately.
- Do not dispose of spent batteries with normal household waste! Take them to the hearing care specialist for proper disposal without harming the environment.

m500 duo ITC / m500 duo ITC P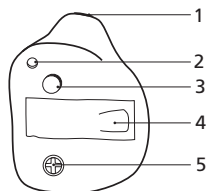


Battery Size 312¹ * (Size original)

1. Wax Filter
2. Ventilation
3. Programmes push button
4. Battery compartment
5. Microphone

¹EN 60086: PR41

m500 duo CIC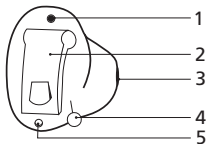


Battery Size 10² * (Size original)

1. Microphone
2. Battery compartment
3. Wax Filter
4. Extractor
5. Ventilation


²EN 60086: PR70

BATTERY (INSERTION AND CHANGE)

1- Open the battery compartment.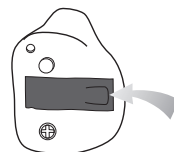

2- Fit the battery and close the battery compartment.

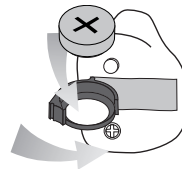
3- To remove the battery, turn the instrument upside down and let the battery fall into your hand.


REMEMBER THAT YOUR HEARING AID GIVES A WARNING OF 2 SOUND SIGNALS 🎵 TO INFORM YOU THAT THE BATTERY IS DEAD.

INSERTING THE INSTRUMENT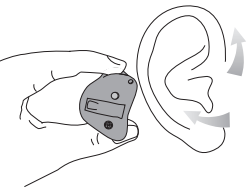

1. Hold your hearing aid with your thumb and your index finger and place it carefully in the ear canal. Move the outer ear slightly upwards or downwards to help.

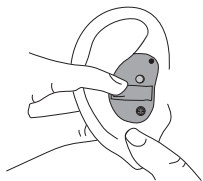
2. Insert the hearing aid completely by pushing very gently. The hearing aid must feel to be properly in place.

REMOVE THE INSTRUMENT


3. To take your hearing aid out, pull the ear lobe to loosen the hearing aid in the ear canal. Use the other hand to open the battery holder and pull gently on the hearing aid to remove it.

NOTE: If your device has an extractor, take hold of the end and pull the hearing aid slowly until it has been removed.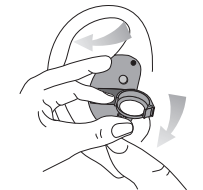


PROGRAMS PUSH BUTTON

Your hearing aid has a push button that enables you to select the desired program.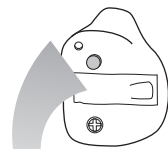

As you press the push button, you will hear tones indicating the programme you are using.

Example: Two tones ♪♪ for program 2.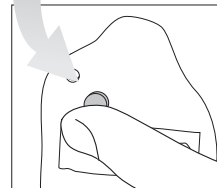


Follow the instructions of your audio prosthetist to get the most out of your hearing aid.

PROGRAMS SELECTION

Each program will permit you to better adapt to different environments or situations of everyday life.

By way of example, your hearing aid may be equipped with the following programs:

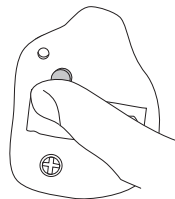- 1- Normal.
- 2- Restaurant.
- 3- Telephone*.
- 4- Traffic.

*Your hearing aid complies with the IEC 60118-13 standard on electromagnetic compatibility.

NOTE

- Clean its ear mould regularly, using a wipe designed for this purpose. The use of MICROCLEAN is recommended (consult your specialist).
- Ask for MICRODRY to prevent the harmful effects of moisture.
- Always have a spare battery available.

ATTENTION!

- Do not clean your hearing aid with alcohol, hydrogen peroxide or detergents: use a soft, moist cloth.

Do not wash your hearing aid!

ENVIRONMENTAL OPERATING CONDITIONS

For optimum operation we recommend using your hearing aids within a range of -20 / 55 °C (degrees Celsius). Likewise, the relative humidity during storage should not exceed 90%.

SYMPTOM	POSSIBLE CAUSE	SOLUTION
The hearing aid does not work	Worn battery defective hearing aid	Replace battery consult your specialist
Insufficient volume	Mould blocked or obstructed	Eliminate obstruction and clean the mould
Sound distorted or unclear	Worn battery	Replace battery

Manufacturer's customer service:

+34 93 300 58 00

helpdesk@microson.es

NOTE

For any other problems or doubts, contact your audioprosthologist or retailer.

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