**microson**

# M-34 Digital

## M-34 DIGITAL FRONT BTE

Printed in Spain

MU12101\_LP2 Rev. B 2013-07-09

Ref. 89611

 MICROSON S.A.  
Pere IV, 160  
08005 BARCELONA SPAIN  
Tel. +34 93 300 58 00  
[www.microson.es](http://www.microson.es)

**microson**

**MANUAL DE USO**

**ES**

**USER'S MANUAL**

**EN**

**MANUALE D'USO**

**IT**

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**PT**

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**FR**

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**Congratulations and welcome!**

You have bought a MICROSON hearing aid featuring the latest digital technology.

Your hearing device is the result of years of research and development and is designed to make sure you enjoy all the sounds of life.

The team at Microson would like to wish you great success with it.

Intended use	16
Disposal Information	16
How it works	17
Explanation of symbols	18
Safety Instructions	19
Description of your hearing aid	20
Battery (insertion and changing)	21
Volume adjustment	22
Switching the instrument ON/OFF	22
Battery Lock	23
Programs push button	24
Audio input	24
Audio Shoe (insertion, extraction)	25
Care and maintenance of your hearing aid	26
Environmental operating conditions	26
Troubleshooting	27

Microson hearing aids are designed for correcting hearing problems and compensating for hearing loss by selective sound amplification. Your hearing aids should be fitted by a hearing care specialist.

For safe use, read the instructions in this manual carefully.


This manual was drawn up in compliance with the EN 1041 and EN 980 standards.

## DISPOSAL INFORMATION

This product must not be disposed together with the domestic waste. All users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a municipal or commercial collection point so that they can be disposed of in an environmentally acceptable manner. Remove the batteries before disposing of the equipment. Do not dispose of old batteries with your household waste, but dispose of them at a battery collection station at a recycling site or in a shop. Consult your municipal authority or your dealer for information about disposal.

Within the European Union, the marked equipment is covered by "Directive 2002/96/EC of the European Parliament and of the Council of 27 January 2003 on waste electrical and electronic equipment." Amended by "Directive 2003/108/EC" (WEEE).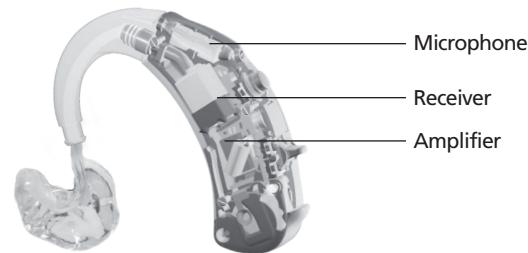

The BTE (Behind the Ear BTE) hearing aids are anatomical instruments placed behind the ear. . Are powered by a battery and are typically designed to cover severe or profound hearing losses. All mechanical and electronic components are contained in an ergonomic housing. They adapt to the user's ear by molds which leaves a conductive tube that attaches to the side of the hearing aid.

**MICROPHONE:** receives the sound and converts it into an electrical signal.

**AMPLIFIER:** digital processor that receives the signal and adapts it to the parameters introduced for each loss.

**RECEIVER:** converts the electrical signal back into sound and transmits it into the user's ear.

Indicates situations where you may be injured.

Indicates unsafe situations or procedures that may lead to injury or damage to your hearing aid.

Indicates possible damage or alteration of the hearing aid or any of its components.

Operating instructions.

Indicates that it is important to follow the advice and warnings contained in this document.

Thanks to CE marking, Microson confirms that its hearing aids comply with European Directive 93/42 EEC concerning medical devices.

This symbol indicates that the products described in these user instructions adhere to the requirements for an application part of Type B of EN 60601-1. The surface of the hearing aid is specified as application part of Type B.

The symbol with the crossed-out garbage bin is to make you aware that this hearing aid may not be thrown away as normal household waste. Please dispose of old or unused hearing aids, at waste disposal sites intended for electronic waste, or give your hearing aid to your hearing care professional for disposal. Proper disposal protects the environment and health.

### WARNING!


- It is dangerous to swallow batteries. Keep batteries out of the reach of children.
- Never use your hearing aids in places where there is medical equipment (X-ray, high frequency, magnetic resonance or short-wave therapy treatments).
- Do not use the hearing aids in areas where there is a risk of explosion!
- **M-34 Digital FRONT BTE** hearing aids can generate output levels in excess of 132 dB<sub>SPL</sub>.

### CAUTION!

- Protect your hearing aids against moisture and sources of heat.
- Do not use your hearing aids while showering, swimming or spraying your hair.
- Protect your hearing aids from hard knocks.
- Never use a microwave or other heat emitters to dry your hearing aids.
- Regular use of a dehumidifier is recommended to keep your hearing aid in optimum working conditions: Ask your hearing care specialist.

### INFORMATION


- Remove the batteries if you do not intend to use your hearing aids for a long time.
- Replace spent batteries immediately.
- Do not dispose of spent batteries with normal household waste! Take them to the hearing care specialist for proper disposal without harming the environment.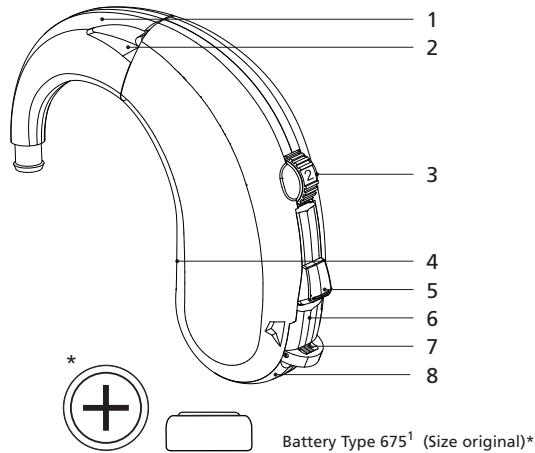

1. Ergonomic-ear-hook design (Standard / Pediatric)
2. Microphone
3. Volume Control
4. Trimmers  
(For use exclusively by the Hearing Aid specialist)
5. Programs push button
6. Audio Input
7. Battery Lock
8. Battery compartment


<sup>1</sup>EN 60086 Designation: PR44

### 1- Opening the battery compartment.

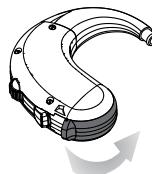
2- Insert the battery with the "+" symbol facing upwards and close the battery compartment.

If the battery is not correctly inserted, the hearing aid will not function.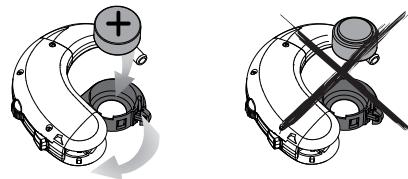

3- To remove the battery, face the opening of the hearing aid downwards, open the battery compartment and allow it to fall into your hand

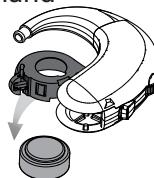REMEMBER THAT YOUR HEARING AID WILL WARN YOU VIA THREE AUDIBLE TONES WHEN THE BATTERY IS DEAD.


The volume control allows you to adjust the amplification of your hearing aid.

Use a finger to rotate the volume control wheel, as indicated in the figure.


Number 1 is the lowest level.  
Number 4 is the highest level.

## SWITCHING ON / OFF



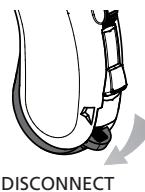
### TURNING OFF:

To turn off your hearing aid, partially open the battery compartment.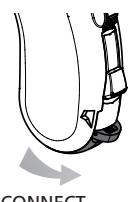

### TURNING ON:

To turn on your hearing aid, close the battery compartment.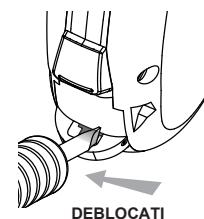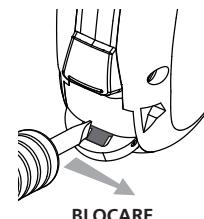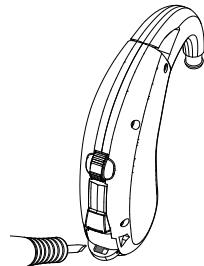

Your hearing aid is equipped with a battery holder locking system. This lock prevents undesired accidental opening of the compartment.

It is also recommended when the hearing aid is for a child, because removal and replacement of the battery must be done under adult supervision. This element is located in the upper portion of the opening tab of the battery compartment.

To prevent opening, insert a small screwdriver and slide it to one side.

Unlocking it can only be done by sliding the safety back to its original position with the help of a small screwdriver.

## PROGRAMS PUSH BUTTON

Your hearing aid has a push button that enables you to select the desired program.


**Position M:** Normal hearing program.

This program is recommended for mobile phone<sup>1</sup> use.

You will hear 1 tone ♪ when you select this program.

**Position T:** Program designed for talking on the phone or for use in public places such as theatres and churches with induction loop. (Consult your specialist).

You will hear 2 tones ♪♪ when you select this program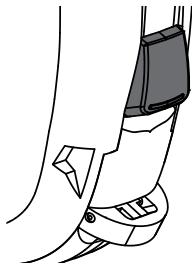

<sup>1</sup>Your hearing aid complies with the EN 60118-13 standard on electromagnetic compatibility.

## AUDIO INPUT

Your hearing aid has a built-in audio input jack. This allows you to connect your hearing aid to other audio equipment with FM system, television or various audio equipment. It transmits the source sound<sup>1</sup> directly to your hearing aid, which keeps the signal clear and free of external interference.


Only use the Audio Shoe designed specifically for the hearing aid.

**⚠ WARNING!**

Risk of injury by electrical shock!


Connect the audio input only to devices that comply with IEC 60065 (IEC standard for audio, video and other electronic devices).

<sup>1</sup>Your hearing aid complies with the IEC 60118-6 norm, which guarantees its compatibility with external electrical and electro-acoustical signal.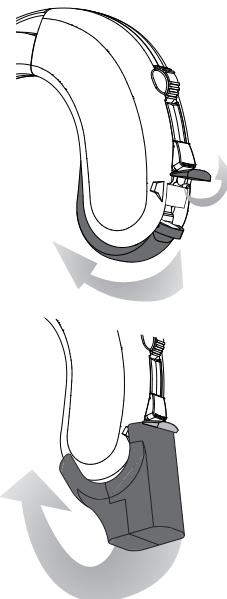

## INSERTION

1. Open the battery door until to the first position, lift the top of the connector and close the battery holder again.
2. Place the audio shoe on top, line up the jack contacts with those that are now visible on the hearing aid.
3. Push the audio shoe downward until it couples with the hearing aid.

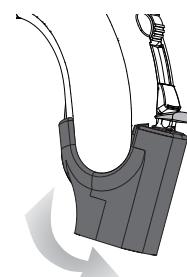## REMOVAL

1. Hold the device with one hand and the shoe with the other.
2. Pull the shoe outwards until it separates from the hearing aid.
3. Close the connector top, hiding the contacts below it.

 NOTE

- Clean its ear mould regularly, using a wipe designed for this purpose. The use of MICROCLEAN is recommended (consult your specialist).
- Ask for MICRODYR to prevent the harmful effects of moisture.
- Always have a spare battery available.

 ATTENTION!

- Do not clean your hearing aid with alcohol, hydrogen peroxide or detergents: use a soft, moist cloth.

Do not wash your hearing aid!

## ENVIRONMENTAL OPERATING CONDITIONS

For optimum operation we recommend using your hearing aids within a range of 0 / 40 °C (degrees Celsius). Likewise, the relative humidity during storage should not exceed 90%.

SYMPTOM	POSSIBLE CAUSE	SOLUTION
The hearing aid does not work	Worn battery defective hearing aid	Replace battery  consult your specialist
Insufficient volume	Mould blocked or obstructed	Eliminate obstruction and clean the mould
Sound distorted or unclear	Worn battery	Replace battery

Manufacturer's  
customer service:  +34 93 300 58 00  
 helpdesk@microson.es

 NOTE

For any other problems or doubts, contact your audioprosthologist or retailer.

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