m34 Digital

m34 Digital ITC
m34 Digital CIC

MU12110 RevA 2010-03-29

Printed in Spain

MANUAL DE USO

ES

USER'S MANUAL

EN

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DE

MANUALE D'USO

IT

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PT

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FR

KULLANIM KILAVUZU

TR

MICROSON S.A.
Pere IV, 160
08005 BARCELONA, SPAIN
Tel. +34 93 300 58 00
www.microson.es

Ref. 81465

microson
SINCE 1958
HEARING AIDS

¡Enhorabuena!

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Congratulations !

You have purchased a MICROSON hearing aid with the latest digital technology.

Your hearing device is the result of years of research and development and has been designed so that you may enjoy the sounds of life once again.

Our team thanks you for your trust and wishes you every success in the future.

This manual has been made in compliance with EN 1041:2008.

Intended Use _____ 12

Safety instructions _____ 13

m34 Digital ITC _____ 14

m34 Digital CIC _____ 14

Battery (Insertion and change) _____ 15

Disconnecting your *m34 Digital CIC* _____ 15

Inserting the instrument in your ear _____ 16

Remove the instrument from your ear _____ 16

Volume adjustment _____ 17

Wax filter cleaning _____ 17

Maintenance and care _____ 18

Troubleshooting _____ 19

Microson hearing aids are intended for correction of hearing problems and hearing loss compensation through selective amplification of sound. Your hearing aid must be fitted by a specialist.

It is important to read and follow the instructions in this manual to avoid injury or damage to your hearing aid.

SYMBOLS		
Pay special attention to the information marked with these symbols		
	WARNING!	Shows situations where could be injured.
	CAUTION!	Shows unsafe situations or procedures that could cause injury or damage to your hearing aid.
	ATTENTION!	Indicates possible damage or deterioration in the hearing aid or any of its components.
	INFORMATION	Indicates that it is important to follow the advice and warnings contained in this document
	CERTIFIES	With the CE mark confirms that your Microson Hearing Aid is in compliance with the European Directive 93/42 EEC for medical devices.

In case of doubt, ask your hearing specialist.

WARNING

- Do not swallow the batteries. Keep out of reach of children.
- Never use your hearing aid in the following places: Areas with medical equipment (X-ray equipment, high-frequency, magnetic resonance or short wave therapy)
- Do not use the headset in areas exposed to danger of explosion!


CAUTION

- Protect your hearing aid from moisture and sources of heat.
- Do not use your hearing aid if you are showering, bathing or spraying your hair.
- Make sure that your hearing aid does not receive any knocks.

INFORMATION

- Remove the batteries when the instrument is not in use over a prolonged period.
- Replace dead batteries immediately.
- Do not throw used batteries into the trash! Give the hearing care professional to be disposed of properly without harming the environment


m34 Digital ITC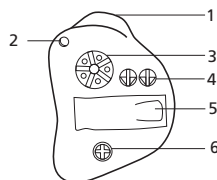

Battery size 312 *Size original

- | | |
|-------------------|--|
| 1. Wax Filter | 4. Trimmers
(For use exclusively by the Hearing Aid specialist) |
| 2. Ventilation | 5. Battery compartment |
| 3. Volume Control | 6. Microphone |


m34 Digital CIC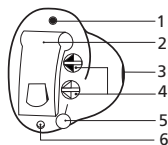


Battery size 10 *Size original

- | | |
|------------------------|--|
| 1. Microphone | 4. Trimmers
(For use exclusively by the Hearing Aid specialist) |
| 2. Battery compartment | 5. Extractor |
| 3. Wax Filter | 6. Ventilation |

BATTERY (INSERTION AND CHANGE)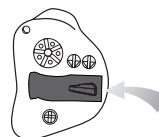

1- Open the battery compartment.

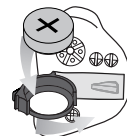2- Fit the battery and close the battery compartment.

3- To remove the battery, turn the instrument upside down and let the battery fall into your hand.

DISCONNECTING YOUR m34 Digital CIC

To disconnect your hearing aid, open the battery compartment.

To connect your hearing aid, close the battery compartment.

INSERTING THE INSTRUMENT

1. Hold your hearing aid with your thumb and your index finger and place it carefully in the ear canal. Move the outer ear slightly upwards or downwards to help.

2. Insert the hearing aid completely by pushing very gently. The hearing aid must feel to be properly in place.

REMOVE THE INSTRUMENT


3. To take your hearing aid out, pull the ear lobe to loosen the hearing aid in the ear canal. Use the other hand to open the battery holder and pull gently on the hearing aid to remove it.

NOTE: If your device has an extractor, take hold of the end and pull the hearing aid slowly until it has been removed.

VOLUME ADJUSTMENT

m34 Digital ITC:


The volume control enables you to connect, disconnect and adjust the amplification of your hearing aid. Use your finger to turn the volume control wheel, as shown in the figure. To switch it off, turn the volume control backwards until it can go no further or until the mark has been reached.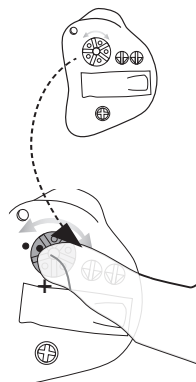

m34 Digital CIC:

The volume control should be adjusted by your specialist.

WAX FILTER CLEANING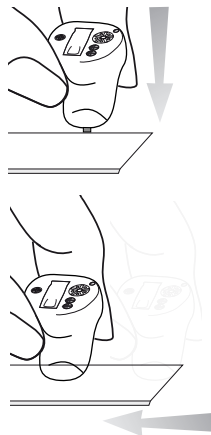


Your hearing aid is equipped with the latest-generation filter for preventing wax filter. Follow the instructions to ensure the most effective maintenance:

Press down lightly, sliding it at the same time, as shown in the figures.

Perform this operation once or twice a day on a suitable surface*.

*(Consult your specialist)

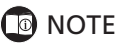NOTE

- Clean its ear mould regularly, using a wipe designed for this purpose. The use of MICROCLEAN is recommended (consult your specialist).
- Ask for MICRODRY to prevent the harmful effects of moisture.
- Always have a spare battery available.

ATTENTION!

- Do not clean your hearing aid with alcohol, hydrogen peroxide or detergents: use a soft, moist cloth.

Do not wash your hearing aid!

SYMPTOM	POSSIBLE CAUSE	SOLUTION
The hearing aid does not work	Dead battery	Replace battery
	Defective hearing aid	Consult your specialist
It is not loud enough	Blocked or dirty waxfilter	Clean dirtiness or changed the waxfilter
Distorted and unclear sound	Spent battery	Replace battery

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