# m6

m6 BTE

MANUAL DE USO

ES

USER'S MANUAL

EN

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DE

MANUALE D'USO

IT

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PT

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FR

KULLANIM KILAVUZU

TK

MUJ7010 Rev.A 2009-10-29

Printed in Spain

Ref. 77359

 **MICROSON S.A.**  
Pere IV, 160  
08005 BARCELONA, SPAIN  
Tel. +34 93 300 58 00  
[www.microson.es](http://www.microson.es)

  
0197

  
microson  
SINCE 1959  
HEARING AIDS

¡Enhorabuena!

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Congratulations !

You have purchased a hearing aid with the latest digital technology.

Your hearing device is the result of years of research and development and has been designed so that you may enjoy the sounds of life once again.

Our team thanks you for your trust and wishes you every success in the future.

This manual has been made in compliance with EN 1041:2008.

|                                                |    |
|------------------------------------------------|----|
| Intended Use .....                             | 14 |
| Safety instructions .....                      | 15 |
| Description of your Hearing Aid .....          | 16 |
| Battery (Insertion and change) .....           | 17 |
| Volume adjustment .....                        | 18 |
| Switching the instrument ON/OFF .....          | 18 |
| Programs push button .....                     | 19 |
| Programs selection .....                       | 19 |
| Auto-Phone .....                               | 20 |
| Automatic activation .....                     | 20 |
| Magnets (Safety warnings) .....                | 21 |
| Maintenance and care of your Hearing Aid ..... | 22 |
| Troubleshooting .....                          | 23 |

Microson hearing aids are intended for correction of hearing problems and hearing loss compensation through selective amplification of sound. Your hearing aid must be fitted by a specialist.

It is important to read and follow the instructions in this manual to avoid injury or damage to your hearing aid.

| SYMBOLS                                                                          |             |                                                                                                                                      |
|----------------------------------------------------------------------------------|-------------|--------------------------------------------------------------------------------------------------------------------------------------|
| Pay special attention to the information marked with these symbols               |             |                                                                                                                                      |
|  | WARNING!    | Shows situations where could be injured.                                                                                             |
|                                                                                  | CAUTION!    | Shows unsafe situations or procedures that could cause injury or damage to your hearing aid.                                         |
|                                                                                  | ATTENTION!  | Indicates possible damage or deterioration in the hearing aid or any of its components.                                              |
|  | INFORMATION | Indicates that it is important to follow the advice and warnings contained in this document                                          |
|  | CERTIFIES   | With the CE mark confirms that your Microson Hearing Aid is in compliance with the European Directive 93/42 EEC for medical devices. |

In case of doubt, ask your hearing specialist.

## WARNING


- Do not swallow the batteries. Keep out of reach of children.
- Never use your hearing aid in the following places:  
Areas with medical equipment (X-ray equipment, high-frequency, magnetic resonance or short wave therapy)
- Do not use the headset in areas exposed to danger of explosion!
- **m6** hearing aids can generate higher output levels up to 132 dB<sub>SPL</sub>.

## CAUTION

- Protect your hearing aid from moisture and sources of heat.
- Do not use your hearing aid if you are showering, bathing or spraying your hair.
- Make sure that your hearing aid does not receive any knocks.

## INFORMATION

- Remove the batteries when the instrument is not in use over a prolonged period.
- Replace dead batteries immediately.
- Do not throw used batteries into the trash! Give the hearing care professional to be disposed of properly without harming the environment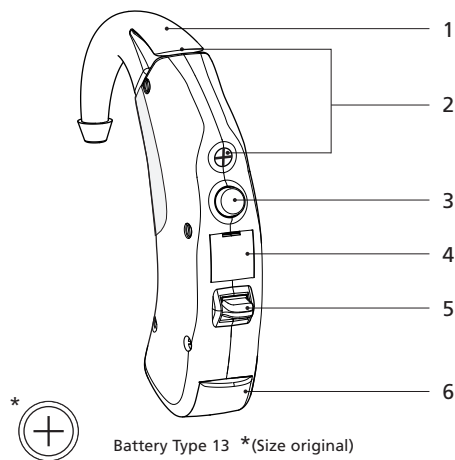

1. Ergonomic-ear-hook design
2. Microphone/s
3. Programs push button
4. Programming connector  
(For use exclusively by the Hearing Aid specialist)
5. Volume Control
6. Battery compartment


1- Open the battery compartment.

2- Fit the battery and close the battery compartment.


3- To remove the battery, turn the instrument upside down and let the battery fall into your hand.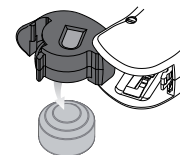

REMEMBER THAT YOUR HEARING AID GIVES A WARNING OF 5 SOUND SIGNALS  TO INFORM YOU THAT THE BATTERY IS DEAD.


## VOLUME ADJUSTMENT

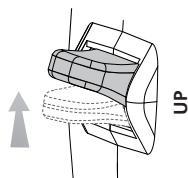
Your hearing aid is totally automatic, but if you prefer manual adjustment, you can use the volume control.

Use your finger to operate the lever up or down, the volume of aircraft go up or down a fraction and a ring tone. Repeat this action until the volume desired.


If you reach the volume limit will warn you with two tones.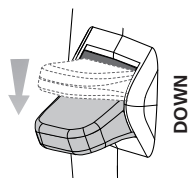

## SWITCHING ON / OFF

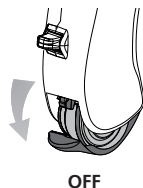To disconnect your hearing aid, partially open the battery compartment.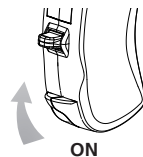

To connect your hearing aid, close the battery compartment.


## PROGRAMS PUSH BUTTON

Your hearing aid has a push button that enables you to select the desired program.

As you press the push button, you will hear tones indicating the program you are using.

Example: Two tones  for program 2.

Follow the instructions of your audio prosthetist to get the most out of your hearing aid.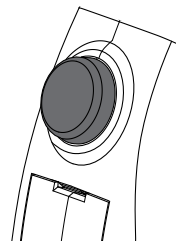


## PROGRAMS SELECTION


Each program will permit you to better adapt to different environments or situations of everyday life.

By way of example, your hearing aid may be equipped with the following programs:

- 1- Normal.
- 2- Restaurant.
- 3- Telephone\*.
- 4- Traffic.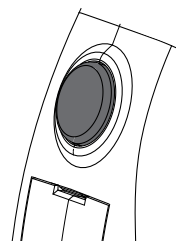

\*Your hearing aid complies with the EN 60118-13:2005 standard on electromagnetic compatibility.

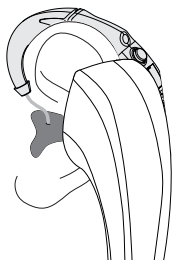Your **m6** is equipped with the **Auto-phone** function that enables your hearing aid to select the telephone program automatically when you put the receiver to your ear.

When you finish the call and remove the phone from your ear, it will revert to the program you were using before.

## AUTOMATIC ACTIVATION

To activate the **Auto-phone** automatically, place the magnetic accessory in your telephone's receiver. Move the magnet gently until it is in the optimum position\* for a reliable connection and a pleasant sound. Next, remove the protector and place the accessory in the receiver.

\*Shaded area of the illustration

### ⚠ CAUTION!

The magnets may affect certain electrical apparatuses. Be careful not to place the magnets near monitors, television sets or other electrical equipment.


Check that there are no credit cards, CDs or DVDs nearby, or other mechanisms sensitive to magnetic fields (minimum 30 cm).

### ⚠ WARNING!

- The magnets may interfere with the operation of medical equipment such as pacemakers and magnetic valves, etc.

Pacemaker users should follow the safety instructions of the pacemaker manufacturer and make use of the magnetic accessory when in no doubt about the safety distance. (Minimum 10 cm)

- Store the magnets out of children's reach. In the event a magnet is swallowed, consult your doctor.

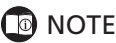NOTE

- Clean its ear mould regularly, using a wipe designed for this purpose. The use of MICROCLEAN is recommended (consult your specialist).
- Ask for MICRODRY to prevent the harmful effects of moisture.
- Always have a spare battery available.

ATTENTION!

- Do not clean your hearing aid with alcohol, hydrogen peroxide or detergents: use a soft, moist cloth.

Do not wash your hearing aid!

| SYMPTOM                       | POSSIBLE CAUSE                  | SOLUTION                                   |
|-------------------------------|---------------------------------|--------------------------------------------|
| The hearing aid does not work | Dead battery                    | Replace battery                            |
|                               | Defective hearing aid           | Consult your specialist                    |
| It is not loud enough         | Blocked or obstructed ear mould | Remove obstruction and clean the ear mould |
| Distorted and unclear sound   | Spent battery                   | Replace battery                            |

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