m6

m6 ITC P DIR

m6 ITC P

m6 CIC P

m6 CIC

MANUAL DE USO

ES

USER'S MANUAL

EN

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DE

MANUALE D'USO

IT

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PT

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FR

KULLANIM KILAVUZU

TK

MU77020 Rev A 2009-10-29

Printed in Spain

 **MICROSON S.A.**
Pere IV, 160
08005 BARCELONA, SPAIN
Tel. +34 93 300 58 00
www.microson.es

0197

Ref. 77360

microson
SINCE 1959
HEARING AIDS

¡Enhorabuena!

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Congratulations !

You have purchased a MICROSON hearing aid with the latest digital technology.

Your hearing device is the result of years of research and development and has been designed so that you may enjoy the sounds of life once again.

The MICROSON team thanks you for your trust and wishes you every success in the future.

This manual has been made in compliance with EN 1041:2008.

Intended Use	14
Safety instructions	15
m6 ITC P DIR	16
m6 CIC	16
m6 CIC P	16
m6 ITC P	17
Battery (Insertion and change)	17
Inserting the instrument in your ear	18
Remove the instrument from your ear	18
Programs push button	19
Programs selection	19
Auto-Phone (Option)	20
Automatic activation	20
Safety warnings Magnets	21
Volume adjustment (Option)	22
Maintenance and care of your Hearing Aid	22
Troubleshooting	23

Microson hearing aids are intended for correction of hearing problems and hearing loss compensation through selective amplification of sound. Your hearing aid must be fitted by a specialist.

It is important to read and follow the instructions in this manual to avoid injury or damage to your hearing aid.

SYMBOLS		
Pay special attention to the information marked with these symbols		
	WARNING!	Shows situations where could be injured.
	CAUTION!	Shows unsafe situations or procedures that could cause injury or damage to your hearing aid.
	ATTENTION!	Indicates possible damage or deterioration in the hearing aid or any of its components.
	INFORMATION	Indicates that it is important to follow the advice and warnings contained in this document
	CERTIFIES	With the CE mark confirms that your Microson Hearing Aid is in compliance with the European Directive 93/42 EEC for medical devices.

In case of doubt, ask your hearing specialist.

WARNING

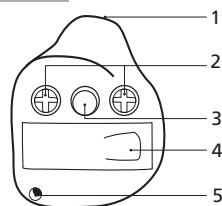
- Do not swallow the batteries. Keep out of reach of children.
- Never use your hearing aid in the following places: Areas with medical equipment (X-ray equipment, high-frequency, magnetic resonance or short wave therapy)
- Do not use the headset in areas exposed to danger of explosion!
- **m6** hearing aids can generate higher output levels up to 132 dB_{SPL}.

CAUTION

- Protect your hearing aid from moisture and sources of heat.
- Do not use your hearing aid if you are showering, bathing or spraying your hair.
- Make sure that your hearing aid does not receive any knocks.


INFORMATION

- Remove the batteries when the instrument is not in use over a prolonged period.
- Replace dead batteries immediately.
- Do not throw used batteries into the trash! Give the hearing care professional to be disposed of properly without harming the environment

Battery Size 312 *(Size original)

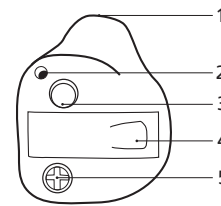
1. Wax Filter
2. Microphones
3. Programmes push button
4. Battery compartment
5. Ventilation

Battery Size 312 *(Size original)

1. Wax Filter
2. Ventilation
3. Programmes push button
4. Battery compartment
5. Microphone

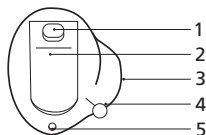
m6 CIC / m6 CIC P

Battery Size 10 *(Size original)

1. Programmes push button
2. Battery compartment
3. Wax Filter
4. Extractor
5. Ventilation

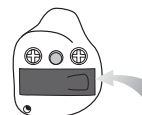BATTERY (INSERTION AND CHANGE)

1- Open the battery compartment.

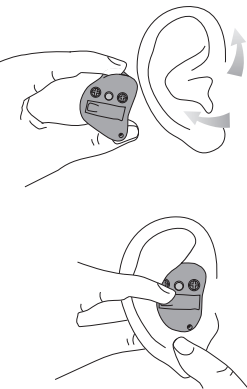
2- Fit the battery and close the battery compartment.

3- To remove the battery, turn the instrument upside down and let the battery fall into your hand.

REMEMBER THAT YOUR HEARING AID GIVES A WARNING OF 5 SOUND SIGNALS  TO INFORM YOU THAT THE BATTERY IS DEAD.


INSERTING THE INSTRUMENT

1. Hold your hearing aid with your thumb and your index finger and place it carefully in the ear canal. Move the outer ear slightly upwards or downwards to help.

2. Insert the hearing aid completely by pushing very gently. The hearing aid must feel to be properly in place.

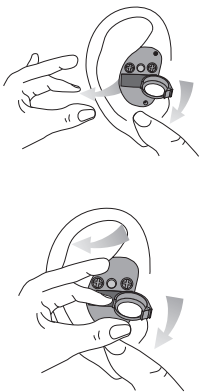REMOVE THE INSTRUMENT


3. To take your hearing aid out, pull the ear lobe to loosen the hearing aid in the ear canal. Use the other hand to open the battery holder and pull gently on the hearing aid to remove it.

NOTE: If your device has an extractor, take hold of the end and pull the hearing aid slowly until it has been removed.

PROGRAMS PUSH BUTTON

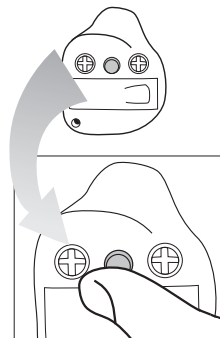Your hearing aid has a push button that enables you to select the desired program.

As you press the push button, you will hear tones indicating the programme you are using.

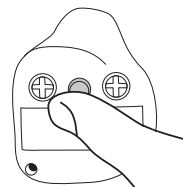
Example: Two tones ♪♪ for program 2.

Follow the instructions of your audio prosthetist to get the most out of your hearing aid.

PROGRAMS SELECTION


Each program will permit you to better adapt to different environments or situations of everyday life.

By way of example, your hearing aid may be equipped with the following programs:

- 1- Normal.
- 2- Restaurant.
- 3- Telephone*.
- 4- Traffic.

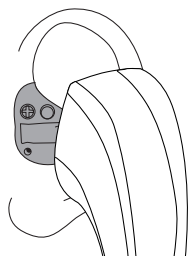*Your hearing aid complies with the EN 60118-13:2005 standard on electromagnetic compatibility.

Your **m6** is equipped with the **Auto-phone** function that enables your hearing aid to select the telephone program automatically when you put the receiver to your ear.

When you finish the call and remove the phone from your ear, it will revert to the program you were using before.

AUTOMATIC ACTIVATION

To activate the **Auto-phone** automatically, place the magnetic accessory in your telephone's receiver. Move the magnet gently until it is in the optimum position* for a reliable connection and a pleasant sound. Next, remove the protector and place the accessory in the receiver.

*Shaded area of the illustration

⚠ CAUTION!

The magnets may affect certain electrical apparatuses. Be careful not to place the magnets near monitors, television sets or other electrical equipment.

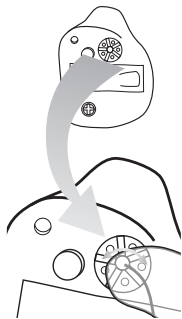
Check that there are no credit cards, CDs or DVDs nearby, or other mechanisms sensitive to magnetic fields (minimum 30 cm).

⚠ WARNING!

- The magnets may interfere with the operation of medical equipment such as pacemakers and magnetic valves, etc.

Pacemaker users should follow the safety instructions of the pacemaker manufacturer and make use of the magnetic accessory when in no doubt about the safety distance. (Minimum 10 cm)


- Store the magnets out of children's reach. In the event a magnet is swallowed, consult your doctor.

The volume control enables you to connect, disconnect and adjust the amplification of your hearing aid.

Use your finger to turn the volume control wheel, as shown in the figure.

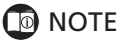MAINTENANCE AND CARE

NOTE

- Clean itsear mould regularly, using a wipe designed for this purpose. The use of MICROCLEAN is recommended (consult your specialist).
- Ask for MICRODRY to prevent the harmful effects of moisture.
- Always have a spare battery available.

ATTENTION!

- Do not clean your hearing aid with alcohol, hydrogen peroxide or detergents: use a soft, moist cloth.

Do not wash your hearing aid!

SYMPTOM	POSSIBLE CAUSE	SOLUTION
The hearing aid does not work	Dead battery	Replace battery
	Defective hearing aid	Consult your specialist
It is not loud enough	Blocked or dirty waxfilter	Clean dirtiness or changed the waxfilter
Distorted and unclear sound	Spent battery	Replace battery

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