microson

MU7071_LP2 Rev. B 2012-09-26

Printed in Spain

M6 BTE

M6 BTE

M6 BTE P

MANUAL DE USO

ES

USER'S MANUAL

EN

MANUALE D'USO

IT

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PT

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FR

Ref. 81566

MICROSON S.A.
Pere IV, 160
08005 BARCELONA, SPAIN
Tel. +34 93 300 58 00
www.microson.es

microson

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Congratulations and welcome!

You have bought a MICROSON hearing aid featuring the latest digital technology.

Your hearing device is the result of years of research and development and is designed to make sure you enjoy all the sounds of life.

The team at Microson would like to wish you great success with it.

Intended use	16
Disposal Information	16
How it works	17
Explanation of symbols	18
Safety Instructions	19
Description of your hearing aid	20
Battery (insertion and replacement)	21
Volume Adjustment	22
Switching ON / OFF	22
Programs push button	23
Programs selection	23
Auto-Phone	24
Automatic Activation	24
Magnets (Safety warnings)	25
Maintenance and care of your Hearing Aid	26
Environmental operating conditions	26
Troubleshooting	27

INTENDED USE

Microson hearing aids are designed for correcting hearing problems and compensating for hearing loss by selective sound amplification. Your hearing aids should be fitted by a hearing care specialist.

For safe use, read the instructions in this manual carefully.

This manual was drawn up in compliance with the EN 1041 and EN 980 standards.


DISPOSAL INFORMATION

This product must not be disposed together with the domestic waste. All users are obliged to hand in all electrical or electronic devices, regardless of whether or not they contain toxic substances, at a local or commercial collection point so that they can be disposed of in an environmentally acceptable manner. Remove the batteries before disposing of the equipment. Do not dispose of old batteries with your household waste, but dispose of them at a battery collection station at a recycling site or in a shop. Consult your local authority or your dealer for information about disposal.

Within the European Union, the marked equipment is covered by "Directive 2002/96/EC of the European Parliament and of the Council of 27 January 2003 on waste electrical and electronic equipment. "Amended by "Directive 2003/108/EC" (WEEE).

HOW IT WORKS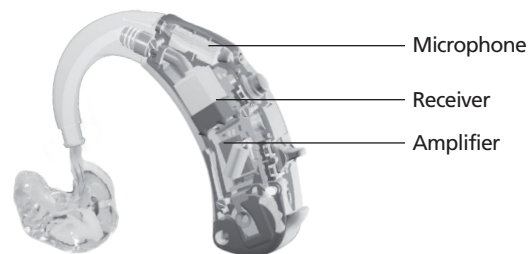


The BTE (Behind the Ear BTE) hearing aids are anatomical instruments placed behind the ear. . Are powered by a battery and are typically designed to cover severe or profound hearing losses. All mechanical and electronic components are contained in an ergonomic housing. They adapt to the user's ear by molds which leaves a conductive tube that attaches to the side of the hearing aid.

MICROPHONE: receives the sound and converts it into an electrical signal.

AMPLIFIER: digital processor that receives the signal and adapts it to the parameters introduced for each hearing loss.

RECEIVER: converts the electrical signal back into sound and transmits it into the user's ear.

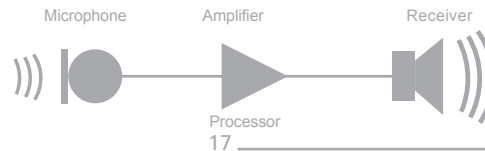Indicates situations where you may be injured.

Indicates unsafe situations or procedures that may lead to injury or damage to your hearing aid.

Indicates possible damage or alteration of the hearing aid or any of its components.

Operating instructions.

Indicates that it is important to follow the advice and warnings contained in this document.

Thanks to CE marking, the manufacturer confirms that its hearing aids complies with European Directive 93/42 EEC concerning medical devices.

This symbol indicates that the products described in these user instructions adhere to the requirements for an application part of Type BF of EN 60601-1. The surface of the hearing aid is specified as application part of Type BF.

The symbol with the crossed-out garbage bin is to make you aware that this hearing aid may not be thrown away as normal household waste. Please dispose of old or unused hearing aids, at waste disposal sites intended for electronic waste, or give your hearing aid to your hearing care professional for disposal. Proper disposal protects the environment and health.

⚠ WARNING!

- It is dangerous to swallow batteries. Keep batteries out of the reach of children.
- Never use your hearing aids in places where there is medical equipment (X-ray, high frequency, magnetic resonance or short-wave therapy treatments).
- Do not use the hearing aids in areas where there is a risk of explosion!
- **M6 BTE / M6 BTE P** hearing aids can generate higher output levels up to 132 dB_{SPL}.

⚠ CAUTION!


- Protect your hearing aids against moisture and sources of heat.
- Do not use your hearing aids while showering, swimming or spraying your hair.
- Protect your hearing aids from hard knocks.
- Never use a microwave or other heat emitters to dry your hearing aids.
- Regular use of a dehumidifier is recommended to keep your hearing aid in optimum working conditions: Ask your hearing care specialist.

📖 INFORMATION

- Remove the batteries if you do not intend to use your hearing aids for a long time.
- Replace spent batteries immediately.
- Do not dispose of spent batteries with normal household waste! Take them to the hearing care specialist for proper disposal without harming the environment.

DESCRIPTION OF YOUR HEARING AID

M6 BTE / M6 BTE P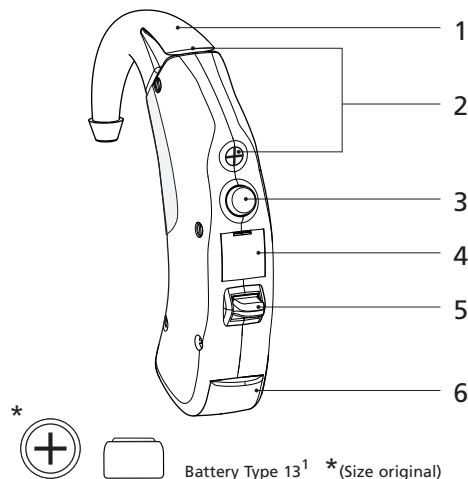

1. Ergonomic-ear-hook
2. Microphone/s
3. Programs push button
4. Programming connector
(For use exclusively by the Hearing Aid specialist)
5. Volume Control
6. Battery compartment



BATTERY (INSERTION AND REPLACEMENT)

1. Open the battery compartment.


2. Insert the battery with the "+" symbol facing upwards and close the battery compartment.

 If the battery is not inserted correctly, the hearing aid will not work.

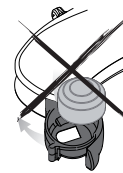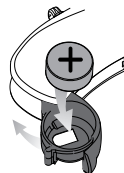
3. To remove the battery, turn the hearing aid upside down, open the battery compartment and allow it to drop into your hand.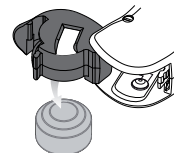

REMEMBER THAT YOUR HEARING AID WILL BEEP 5 TIMES  TO LET YOU KNOW WHEN THE BATTERY IS SPENT.

VOLUME ADJUSTMENT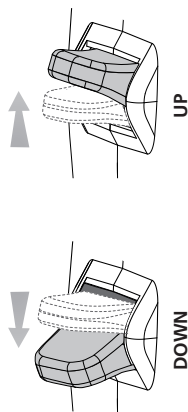


Your hearing aid is totally automatic, but if you prefer manual adjustment, you can use the volume control.


Use your finger to operate the lever up or down, the volume of aircraft go up or down a fraction. Repeat this action until the volume desired.

If you reach the volume limit will warn you with one tone.

SWITCHING ON / OFF

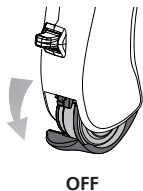
To disconnect your hearing aid, partially open the battery compartment.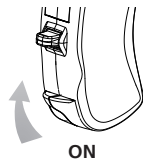

To connect your hearing aid, close the battery compartment.

PROGRAMS PUSH BUTTON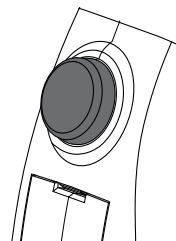


Your hearing aid has a push button that enables you to select the desired programme.

As you press the push button, you will hear tones indicating the programme you are using.

Example: Two tones ♪♪ for programme 2.

Follow the instructions of your audio prosthetist to get the most out of your hearing aid.

PROGRAMS SELECTION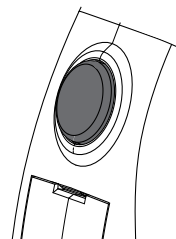


Each programme will permit you to better adapt to different environments or situations of everyday life.

By way of example, your hearing aid may be equipped with the following programmes:

- 1- Normal.
- 2- Restaurant.
- 3- Telephone*.
- 4- Traffic.

*Your hearing aid complies with the EN 60118-13 standard on electromagnetic compatibility.

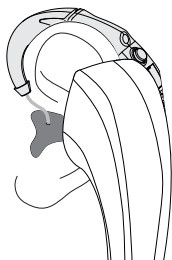Your **M6 BTE / M6 BTE P** is equipped with the Auto-phone function that enables your hearing aid to select the telephone program automatically when you put the receiver to your ear.

When you finish the call and remove the phone from your ear, it will revert to the program you were using before.

AUTOMATIC ACTIVATION

To activate the Auto-phone automatically, place the magnetic accessory in your telephone's receiver. Move the magnet gently until it is in the optimum position* for a reliable connection and a pleasant sound. Next, remove the protector and place the accessory in the receiver.

⚠ CAUTION!

The magnets may affect certain electrical apparatuses. Be careful not to place the magnets near monitors, television sets or other electrical equipment.

Check that there are no credit cards, CDs or DVDs nearby, or other mechanisms sensitive to magnetic fields (minimum 30 cm).

⚠ WARNING!

- The magnets may interfere with the operation of medical equipment such as pacemakers and magnetic valves, etc.

Pacemaker users should follow the safety instructions of the pacemaker manufacturer and make use of the magnetic accessory when in no doubt about the safety distance. (Minimum 10 cm)

- Store the magnets out of children's reach. In the event a magnet is swallowed, consult your doctor.

NOTE

- Clean its ear mould regularly, using a wipe designed for this purpose. The use of MICROCLEAN is recommended (consult your specialist).
- Ask for MICRODRY to prevent the harmful effects of moisture.
- Always have a spare battery available.

ATTENTION!

- Do not clean your hearing aid with alcohol, hydrogen peroxide or detergents: use a soft, moist cloth.

Do not wash your hearing aid!

ENVIRONMENTAL OPERATING CONDITIONS

For optimum operation we recommend using your hearing aids within a temperature range of 0 / 40 °C (degrees Celsius). Likewise, the relative humidity during storage should not exceed 90%.

SYMPTOM	POSSIBLE CAUSE	SOLUTION
The hearing aid does not work	Expired battery defective hearing aid	Replace battery consult your specialist
Insufficient volume	Mould blocked or obstructed	Eliminate obstruction and clean the mould
Sound distorted or unclear	Expired battery	Replace battery

Manufacturer's
customer service:

+34 93 300 58 00
helpdesk@microson.es

NOTE

For any other problems or doubts, contact your audioprosthologist or retailer.

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